## **Treefy** Patient Screen - Column Headers

Last Modified on 02/23/2018 11:52 am EST

You change the information shown on the Patient Screen by selecting the Settings cog in the upperright corner of your screen. The currently selected options are the default settings, however you can select, deselect and rearrange the order of your column headers.


Patient Lis	st Settings	×
Table Colu		
📀 Accoun	t Updated Date & Time	2
📀 🛛 Full Nar	ne 🕒 Add	
📀 Patient	ID	
Patient	DOB	
📀 Studies		
First Na	me	
Last Na	me	
Middle I	Name	
Gender		
Created	Date & Time	
* drag to cha	nge order	

Selecting an active column header will deselect that option and remove it from your Patient screen, whereas selecting a non-active column header will activate it.

The order that they are listed on the pop-up window is the order that they will appear on the Patient screen. To change the order, click and hold your mouse button (or touch and hold if you are on a tablet) and drag the option to a new location.

In the example below, Account, Full Name, Patient ID and Patient DOB were deselected, and Last Name and First Name were selected and dragged to the top of the list. Add was also selected (we will explain this more below).

## Trcefy

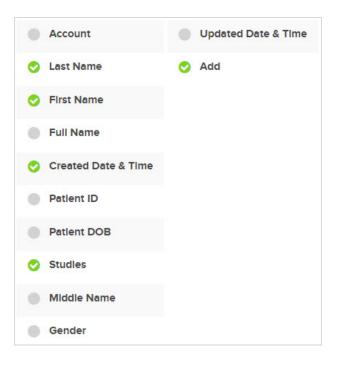

Our new columns look like this:

Last Name	First Name	Created Date & Time	Studies	Add
Thurman	Gerald	2017-03-16 2:59 PM MST	Show Studies (1)	+

• Selecting the + button in the Add column opens a pop-up window for creating a study that does not have DICOM, such as a study that is just a report.

## See instructions for creating a study without DICOM

- If we select the Last Name column header, it will order our patients in alphabetical order (A-Z) by last name
  - If we select column header again, it will reverse the order (Z-A)
- If we select **Created Date & Time**, patients will be added from the newest patient to have a study in Tricefy, to the oldest patient.

You will know how studies are listed because the selected header will be bold. In the example below, **Last Name** is selected so patients will be listed alphabetically by last name:

Last Name ~	First Name	Created Date & Time	Studies	Add
-------------	------------	---------------------	---------	-----

## Saving your Settings

Changes to your settings are automatically saved and will be used the next time you log into Tricefy. Changes on this screen do not affect other members of your clinic.



You can return to the to default settings (shown in the first screenshot) by selecting **Reset Settings** within your Personal Settings.

C Learn more about your Personal Settings