



Automation - Automatically Create Study PDF

Last Modified on 07/19/2018 3:55 pm EDT

Automation rules automatically process images based on data (email addresses, names, labels, etc.) entered into the imaging system prior to an examination. Tricefy uses this information to automatically send studies to patients, apply labels, and other automated actions.

 [Instructions on creating a new automatic process](#) .

When creating a process to automatically create a PDF of all study images, choose **Automatically Create and Attach a PDF report of the Study** as the automation type. Contents of this study are determined by the settings within the automation rule and include:


- Summary page (see example below) with a link to the study
- Study images (thumbnails)
- The first frame of any cine clips
- Any reports or other PDF documents attached to the study (
 - These documents will be placed at the end of the PDF

After the new process has been created, the configuration screen displays.

First, select **Add New Condition** to dictate what triggers the automated process. It is important to note that a condition is not required for creating an automatic process. If conditions are not added, the automatic process is always applied.

Select the red **X** to remove a condition.

If adding a condition, select an option under the **When** drop-down menu to specify when the automation is applied:

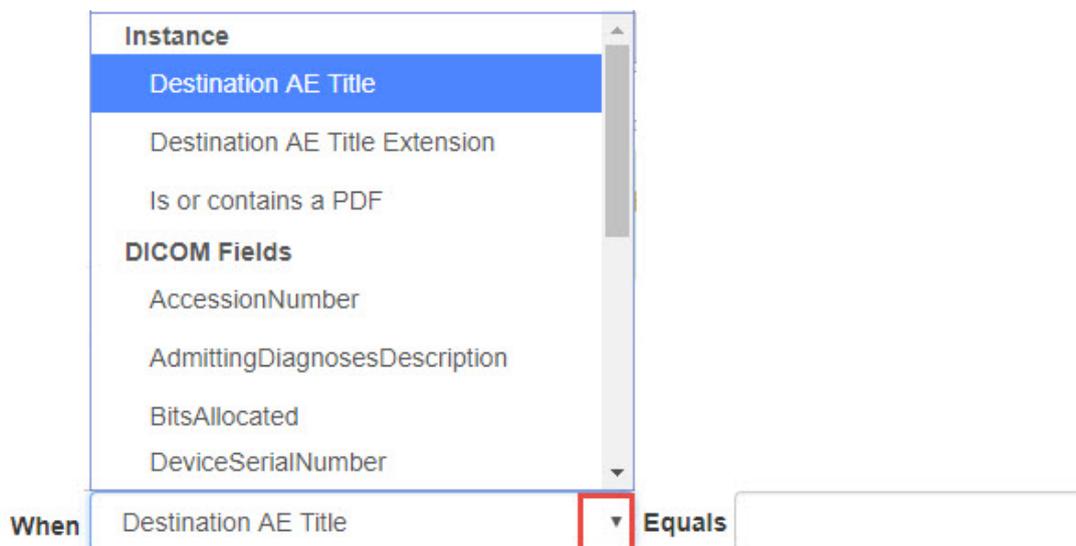

- Choosing an option under **Instance** will automatically create PDFs for the following instances:

- Destination **AE Title** or **Extension** will create a PDF each time a specific imaging system is used
 - We created two stores when configuring the ultrasound system
 - These stores are typically named "archive" and "patient" - [refer to your DICOM setting:](#)
 - Enter the name of the store into the Equals text box
- **Is or contains a PDF** will create a PDF each time the study contains a PDF document
 - Leave the Equals text box empty
- Choosing an option under **DICOM Fields** will create a PDF each time specific information is entered into that DICOM field
 - Type the information that needs to be entered into the Equals text box

Next, complete the following settings:

Wait time before generating PDF

15 minutes ▾

Wait this long for additional files before generating the PDF.

Include Images
Adds image files to the PDF.

Include Secondary Captures
Add secondary capture type images to the PDF.

Append PDFs
Add all PDF files in the study as additional pages in the PDF.

Use Anonymous Link
In place of a permalink

Secondary Captures Use Full Page
Reports from the modality may not be legible as quarter-page sized thumbnails

[Update Automation Settings](#)

- **Wait time:** This is amount of time between when the study is uploaded to Tricefy and when the PDF is generated
 - **15 minutes** is the recommended wait time for most systems
 - Select **30, 45,** or **60** minutes if your system is slower or integrated with an EMR
 - Select **Instant** if you would like the PDF made without any delay
- Uncheck **Include Images** to create a PDF without any images
 - PDF will only contain a summary page (see example below)
- Uncheck **Include Secondary Captures** to exclude items that are not images or PDFs
 - This includes SR reports and images exported from an EMR or other system
- Check **Append PDFs** to include any PDFs attached to the study at the end of the document
- Check **Use Anonymous Link** to replace the permanent link (default) with an anonymized patient link
 - Use this option if intending to share the PDF with patients


- If including secondary captures, we recommend leaving **Secondary Captures Use Full Page** checked
 - Secondary captures often contain text, which is hard to read within a thumbnail

Select **Update Automation Settings** to save the content on this screen and return to the [Automation Setup](#) page.


This process is now listed under **Saved Automations**, where it can be edited, deleted, or disabled.

Sample PDF

The PDF will automatically generate once all files are uploaded and the wait time has elapsed. You will see your PDF within the study:

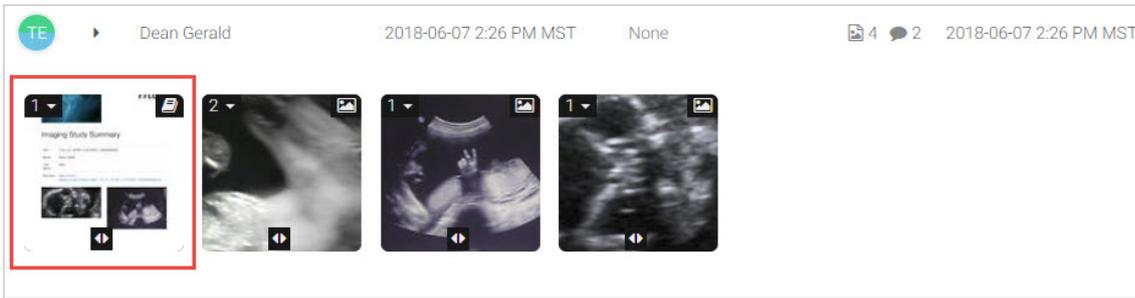

Select the newly-created report to display it in the [viewer](#) :

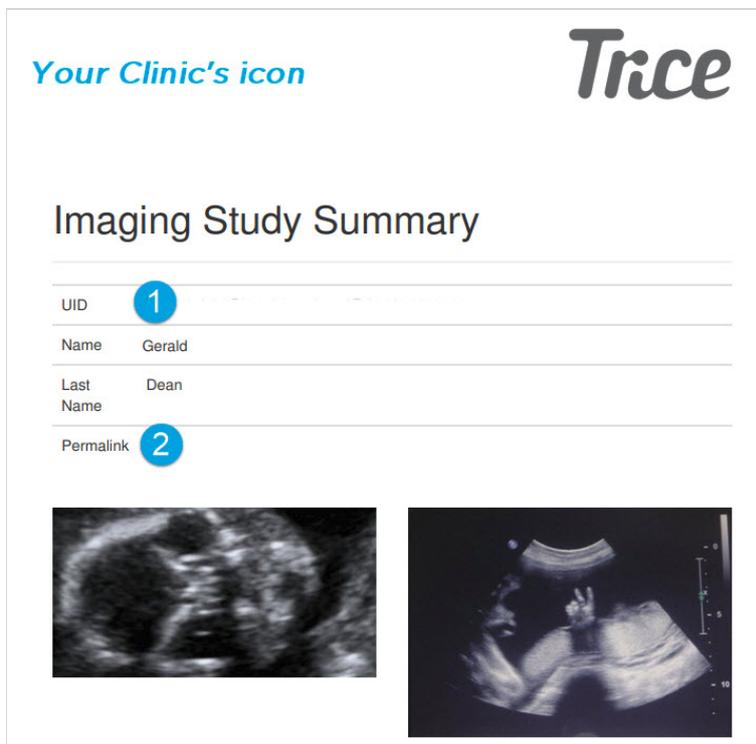

The PDF will have a cover a page that includes the patient's name, thumbnails (if **Include Images** is selected in the automation settings), as well as:


- 1 A Unique ID that is applied to every study

2 A link to the study

- If **Use Anonymous Link** is selected in the automation settings, the link will be a [patient link](#) with all identifiable information removed
- If Use Anonymous Link is not selected in the automation settings, the link will open the study using the [Tricefy viewer](#)

Sources

Selecting the automated PDF within your study while having the right-side info panel opened, will always have the Source (AE Title) as "automation" and Destination (AE Title) as "study-pdf":

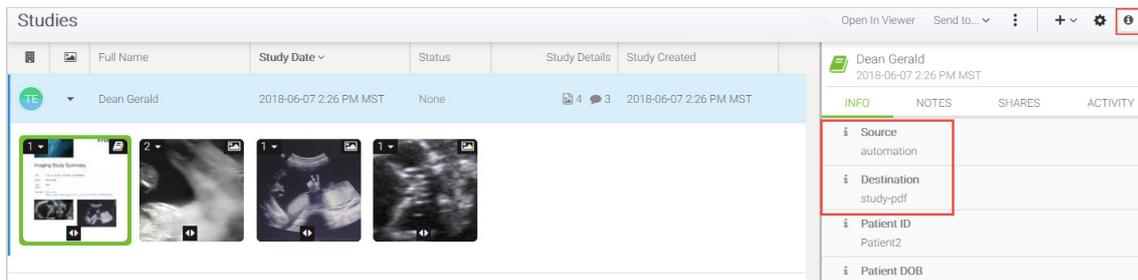

You can use these sources to create additional automation rules, such as [automatic labeling](#) .

Automatic Download

If you would like to automatically download your study PDFs, congratulations, you are already done with Step 1! To proceed:

2) Install the Uplink for downloading content from Tricefy

 [Learn more and view instructions](#)

3) Create a local study route (within your [Account Settings](#)) to tell the new Uplink where to export your PDFs (this will be a location on your local network)

 [Instructions for local routing](#)

4) Create a second automation rule to automatically download study PDFs as soon as they are generated

 [Instructions for automatic download](#)