



Send to EMR

Last Modified on 12/03/2019 4:19 pm EST

You can send and resend Tricefy links to Athena or Allscripts by selecting the study (or studies) on your [Study List](#) and choosing **Send to EMR** from the [Send to Menu](#):

The screenshot shows a table titled "Studies" with columns: Full Name, Patient ID, Patient DOB, Study Date, Study Details, and Sonographers. Three rows are visible, each with a Tricefy icon and a dropdown arrow. The dropdown menu for the first row is open, showing options: "Send to Patient", "Send to Doctor", "Send to DICOM Server", and "Send To EMR". A mouse cursor is pointing at the "Send To EMR" option.

	Full Name	Patient ID	Patient DOB	Study Date	Study Details	Sonographers
TE	June Iodie	5960		2019-04-12 6:22 PM PDT	1 2	
TE	Farmer Carol	1.1.1.1.1.3		2019-04-02 11:05 AM PDT	40 3 3 54 1	
TE	Smith Sally	1.1.1.1.1.8		2019-03-21 11:27 AM PDT	27 2 2 39	

Athenahealth users will see the following pop-up window:

The "Send to Athena" window has a "Provider" dropdown menu set to "Delilah, Heythere (ID: 27)". Under the "Send" section, there are two checkboxes: "Imaging Results Hyperlink" (unchecked) and "Choose from PDFs" (checked). Below these are buttons for "Clear", "Select Study PDFs", and "Select Report PDFs". A preview area shows three study thumbnails. At the bottom left, it says "2 studies selected". At the bottom right are "Cancel" and "Send" buttons.

- Select the **Provider** from the Provider Dropdown menu
- Check **Imaging Results Hyperlink** to send an exam link to the patient chart (the link will open in the exam in the [Tricefy Viewer](#))
- Check **Choose from PDFs** to view and choose specific PDFs to send to the patient chart
- Check both boxes to send an exam link and specific PDFs to the patient chart

Select the **Send** button when you are done making your selections.