


Moving Studies between Clinic Accounts

Last Modified on 2020-11-09 13:31

Note: This feature is only applicable to users of multiple clinics.

Members of multiple clinics can transfer studies within accounts by selecting the study (or studies) and choosing **Move** from the action menu at the top of the screen:

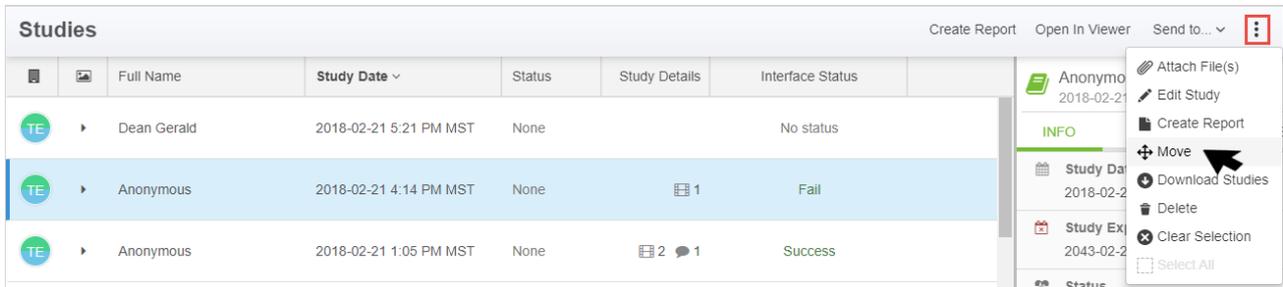

A pop-up window displays:



- Select the Clinic Account to receive the study from the available options in the **Destination Account** menu
- Check **Copy instead of Move** to add a clone of the study in the Destination Account while leaving the study in the current account as well
 - If this box is not selected, the study will be removed from the current account and only located in the new account
- Check **Anonymize all Image Data** to remove identifiable information (patient name, birth date, etc.) from all images
 - If the study contains non-DICOM files (such as a PDF report), an error will display as these cannot be anonymized
- Select **Move Studies** to complete this action; the study will display in the Study List of the Destination Account