























## Icon Dictionary

Last Modified on 02/26/2018 6:25 pm EST

	Toggles the side (information) panel - A square outlines the icon when opened
	Shown on Study List (Details column) - Study contains X number of images
	Shown on Study List (Details column) - Study contains X number of cine clips
	Shown on Study List (Details column) - Study was sent to patient X number of times
	Shown on Study List (Details column) - Study has X number of notes/annotations
	Shown on Study List (Details column) - Study has X number of consultations
	Shown on Study List (Details column) - Study contains X number of reports (New feature under development)
	Shown on Consult Screen - Consultation has X number of collaborators
	Shown on thumbnails in Study List - Select icon to re-arrange ( <a href="#">drag-and-drop</a> ) the order of thumbnails
	Shown at top of Viewer - Select icon to add annotation (icon is <a href="#">blue</a> when activate)
	Shown at the top of Viewer (Active by default) - Allows image to be re-positioned (icon is <a href="#">blue</a> when active)
	Study menu for sharing, downloading, and exporting studies
	Shown on thumbnail - file is an image
	Shown on thumbnail - file is a 3D image
	Shown on thumbnail - file is a cine clip
	Shown on thumbnail - file is a report (new feature under development)
	Shown on thumbnail - file is a document
	Shown on thumbnail - file includes X number of notes and/or annotations
	Opens the User Menu (top-right corner of the screen) for making changes to settings, accessing Help, and logging off
	Uplink Status * - Uplink is not installed
	Uplink Status * - Uplink is not online (Check your internet connection) - <a href="#">See more trouble trouble shootings steps</a>
	Uplink Status * - Uplink is functioning properly

\* If Uplink Status is not shown at the top of the screen, check your [Personal Settings](#) to confirm the option is selected