

Viewer: Overview

Last Modified on 2021-10-20 13:45


The Tricefy Viewer provides a colorful and up-close look at all your images, clips, reports, and attachments.

Open any study or thumbnail in the viewer by either


- Double clicking a study or thumbnail on the Consults or Studies screens
- Single clicking a study on the Studies and selecting the **Open in Viewer** button at the top of the screen
- Accepting an invitation to a consult (the shared study will open in the viewer)

Navigating the Viewer

The top of the Viewer, shown below in the **red** box, contains most of the functionality:

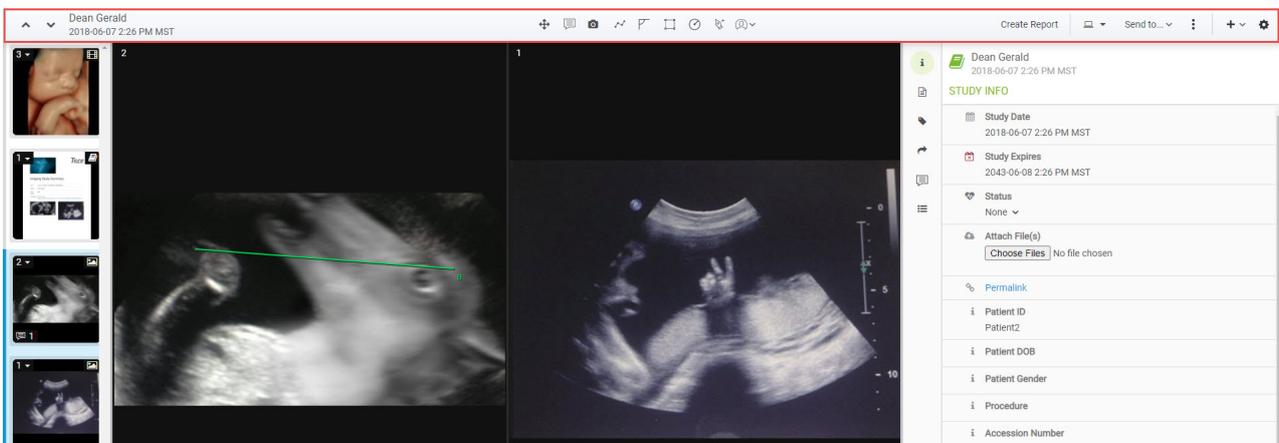

- The   buttons scroll to the next and previous studies on your [Studies List](#)
- Thumbnails selected on Studies screen remain selected in the Viewer. This means that if you share the study with a patient or colleague, they will only receive the selected images. Selected thumbnails will have a **green** outline.
 - To select or unselect specific thumbnails, return to your Studies screen
 - To select all images within the study, choose **Select All** from the action menu at the top of the screen:

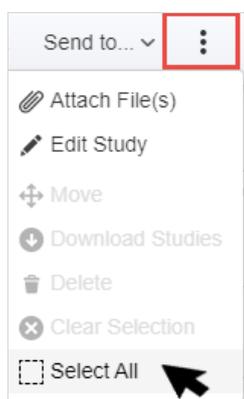

[Learn more about the Action Menu](#)


Move Cursor and Annotation Tools

The move cursor at the top-center of your screen moves the image around the quadrant. Double-clicking the image enlarges the image to full capacity. Select the move cursor again to return the image to its original size.

Annotation tools (available to those with permissions) allow you to add lines, shapes, notes and to view measurements.

Screen Display

The monitor icon at the top of the screen specifies the number of images to display, where to display them, and toggles the thumbnails sidebar:

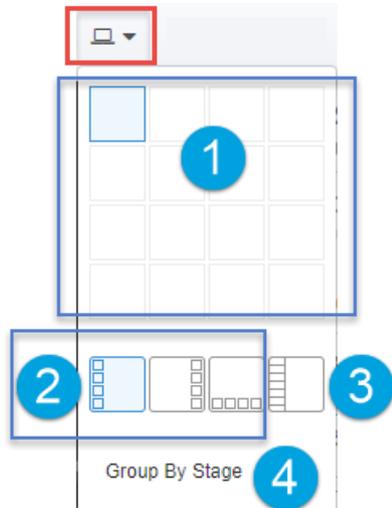


- 1** Highlight the number of quadrants to display (horizontal and vertical)
 - Highlight just one box to view a single image
- 2** Select where you would like to display the thumbnails
 - The active option is blue; select the active option to disable thumbnails
- 3** Toggle the study list sidebar
 - This icon is blue when active
- 4** Uses DICOM tags (StageName, NumberOfStages, StageNumber) to put images and clips side-by-side


Below are some examples of possible screen layouts:



One image, thumbnails on bottom

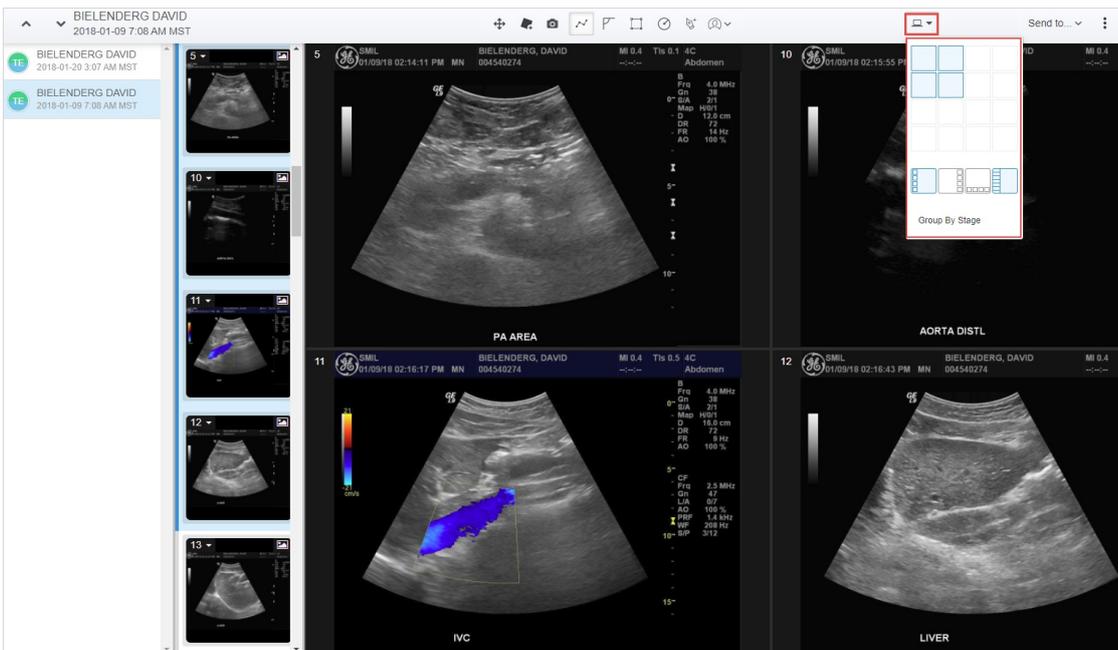

4 images, study list active, thumbnails on left

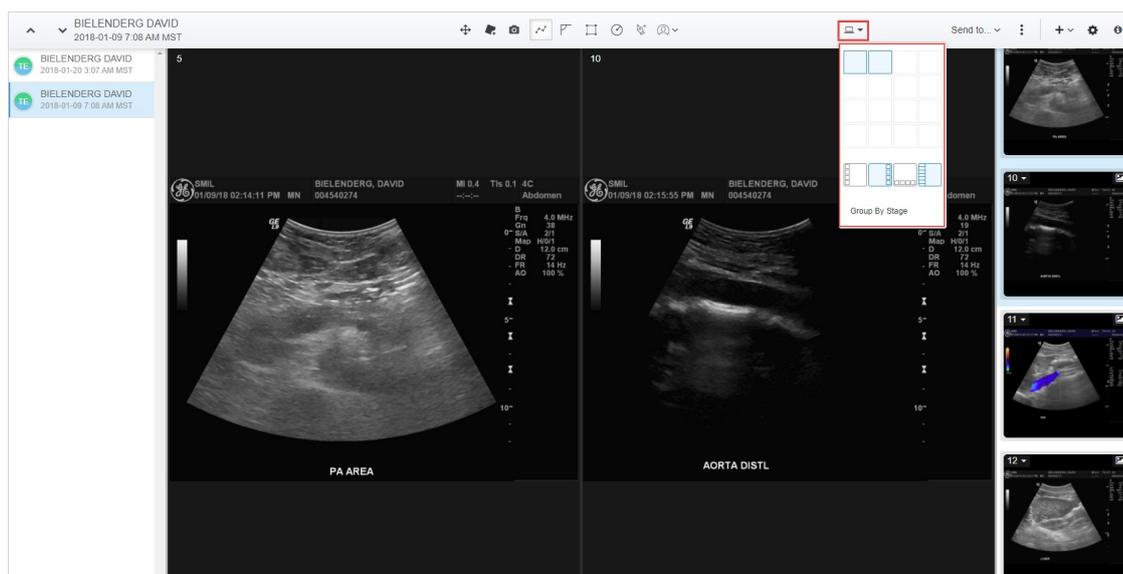

2 images, study list active, thumbnails on right

 See additional instructions for modifying the Viewer Page Settings and Layout

Note: Viewing thumbnails and the Study List is not possible when comparing studies