


Viewer: Page Settings and Layout

Last Modified on 2021-03-18 18:40


The Settings Cog at the top of the Viewer screen displays page settings:

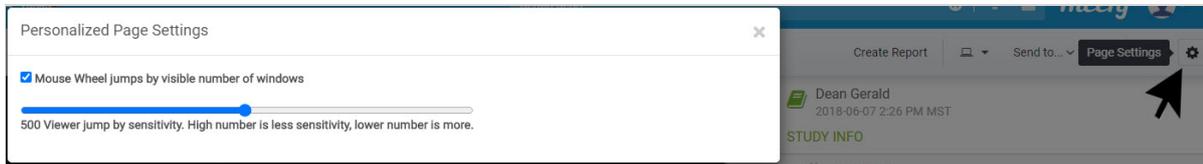

- **Mouse Wheel** checkbox toggles the scrolling functionality of your mouse wheel:
 - When the Mouse Wheel checkbox is selected, scrolling the wheel up and down will scroll up and down through your images
 - Slide the scroll setting **right to increase** sensitivity (scroll faster)
 - Slide the scroll setting **left to decrease** sensitivity (scroll slower)
 - Mouse wheel:

Monitor Settings

Selecting the  icon at the top of the Viewer allows you to control how many images to display. You can display up to 16 images at once (4 x 4) by highlighting the number of squares (shown below in the red box) to represent how you would like them to appear in the Viewer.

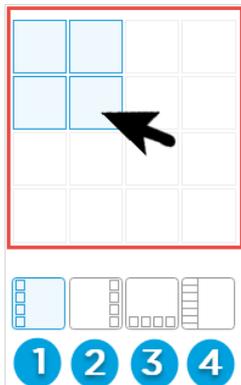

The last icons boxes (labeled above 1-4), toggle the Thumbnail and Study List sidebars:


- 1 Displays Thumbnails on the right side of the screen
 - 2 Displays Thumbnails on the left side of the screen
 - 3 Displays Thumbnails at the bottom of the screen
 - 4 Displays your Study List on the right side of the screen
- A **blue** icon means the sidebar is active; select the icon to turn off the sidebar
 - A **white** icon means the sidebar is not active; select the icon to turn on the sidebar

Examples


The examples below show how different setting variations appear in the viewer.



1 image, thumbnails on bottom

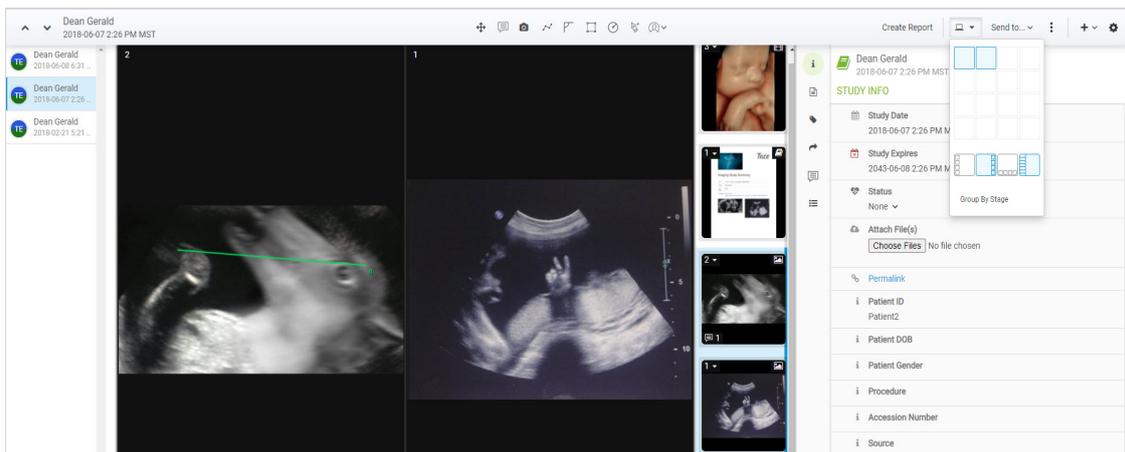


2 images, study list active, thumbnails on right, and side panel open

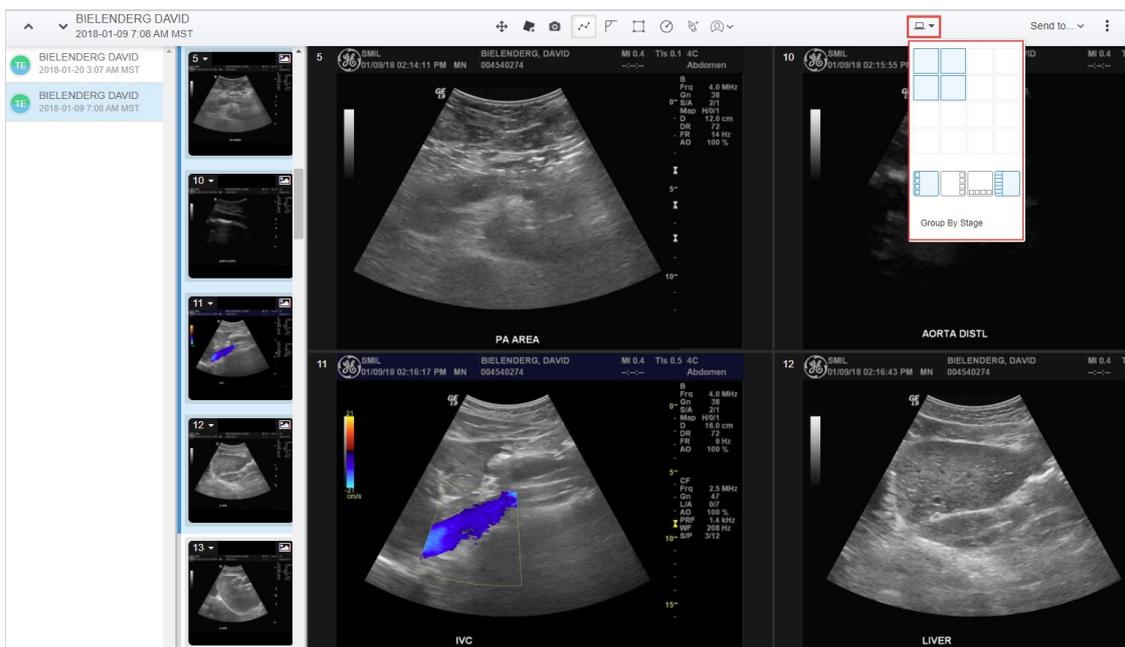

4 images, study list active, thumbnails on left



2 images, study list active, thumbnails on right

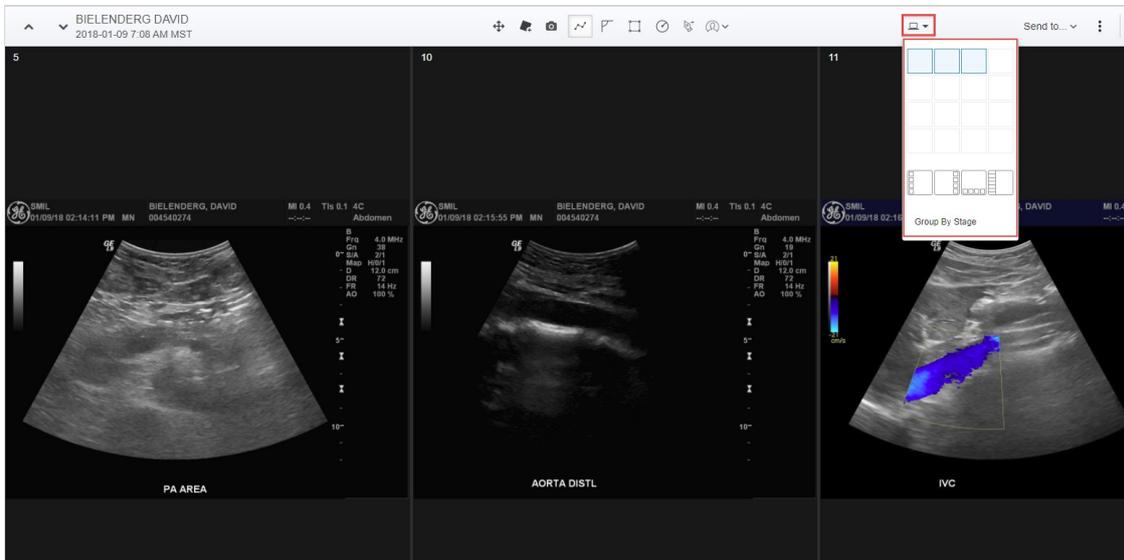

3 images, side-by-side, without any sidebars



6 images, thumbnails on right



Note: The viewer will display thumbnails in consecutive order. To view specific thumbnails side-by-side, [change the order on your Study List](#).
