

Viewer: Thumbnails

Last Modified on 2021-03-18 18:44

Thumbnails are shown in the [Viewer](#) when the sidebar is activated using the monitor settings () at the top of the screen:




 [Instructions for activating your sidebar](#)

Thumbnails are marked with the following icons:

- file is an image
- file is a 3D image
- file is a cine clip. These play automatically when opened in the Viewer
- indicates the number of notes and annotations associated with a file
- thumbnail is selected (it will be included if the study is shared)

displays the following menu, specific to the individual thumbnail:

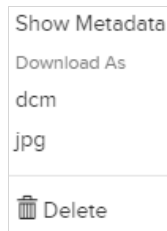

- **Show Metadata:** Displays embedded data that is used for automation. Account Administrators can configure automation within the Account Settings
- **dcm:** Downloads thumbnail as a DICOM file. If the file is not DICOM, this option is not available
- **Jpg:** Downloads thumbnail as a JPEG file
- **Delete:** Removes file from the study.

*Note: Cine clips include the option to download as an **MP4** file; reports and PDF documents include option to download as **Application/PDF** file.*
