


Adjusting User Profile Settings

Last Modified on 2026-03-18 13:05

To access your settings, select **Personal Settings** from the user menu at the top of the screen:

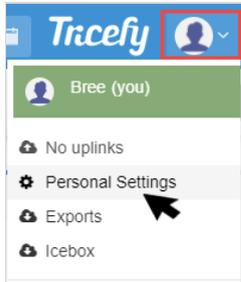

The first tab (default tab) contains Profile Settings. Select **Back to Tricefy** to return to the Studies screen:

Profile [Change Password](#) [Memberships](#) [Notifications](#) [Two-Factor](#)

Profile

Email address

This is used for logging into Tricefy.

Display name

Phone number

This is used for receiving notifications.

Display language

 ▼

Default account

 ▼

Select the account that you want to see in the study list after you login.

Default account for new uplinks

 ▼

Select the account that you want new scanners to send to by default.

Show Uplink status

Check this to display the Uplink Status in the top menu bar.

Signature

No file chosen

- **Email Address:** This is the email address used to log into Tricefy



- If you change your email address, you will receive an email notification to the [previous](#) email that your login email was updated
- **Display Name:** This is the name that appears throughout Tricefy, including activity logs and Tricefy reports
- **Phone Number:** This is the number that receives text notifications if you have SMS notifications enabled
- **Display Language:** Changing this option will change the language shown on your screen
- **Default Account:** This is the account that displays when logging into Tricefy. Selecting **All Accounts** will display patients, studies, and consults belonging to all your clinic accounts.
- **Default Account for New Uplinks:** This is the account that will receive images for any newly installed uplinks
 - **Auto** is default (and recommended) option - studies will be sent to the account that installed the new uplink
- **Show Uplink Status:** When this option is checked, your Uplink status will be shown at the top of the user menu to indicate if your Uplink is working properly
 - A **gray** icon indicates that Uplink is not installed on the computer/server
 - A **green** icon indicates Uplink is functioning properly
 - An **orange** icon indicates that some Uplinks are working but at least one Uplink is not
 - This is only applicable to users with multiple Uplinks
 - A **red** icon indicates Uplink is installed but not functioning. This is often because the computer is turned off or disconnected from the internet.
 - If your Uplink status is **orange** or **red**, [refer here for additional troubleshooting steps](#)

Click "Save Settings" to apply the changes

Signature: Used for signature insert on finalized Tricefy reports

- This is only applicable to Tricefy report templates with the "Signature" element in it
 - The signature must be an image file (JPG or PNG)
-

Reset Settings

The **Reset Settings** button returns all settings to the original, default values. These include:

- Language set to **English**
- **Uplink Status shown** in user menu
- **Email Notifications** turned on: [See instructions on turning off notifications](#)
- Viewer displays **four images** (4 x 4) and **study list sidebar active** (monitor settings)
- Page Columns on your **Studies** screen returned to **Full Name, Patient ID, Patient DOB, Study Date, Study Details, Sonographers, and Performing Physician.**
- Page Columns on your **Patients** screen returned to **Full Name, Patient ID, Patient DOB, and Show Studies**

 [Learn more about changing columns on the Studies screen](#)

 [Learn more about change columns on the Patients screen](#)

A confirmation message displays before the settings are reset. Select **OK** to proceed. These settings do not need to be saved and this action cannot be undone.
