


Uplink Appears Offline

Last Modified on 2021-09-10 14:14

Uplink is the software that transfers data from your ultrasound system to the Tricefy cloud. In order to transfer data, Uplink requires an internet connection. If this connection is lost, the Uplink status will appear offline or not installed.

By default, the Uplink status is shown at the top of the user menu. If you don't see the status in your menu, [go to your Personal Settings](#) and select the [Show Uplink Status](#) checkbox.

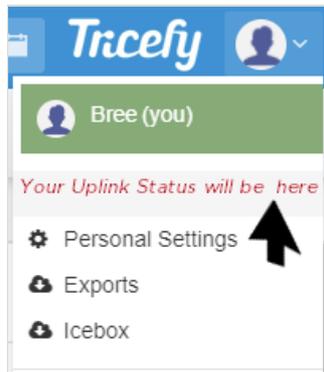

- A **green** icon means that your Uplink is functioning properly
- An **orange** icon means that your Uplink is installed but not online
- A **gray** icon means that Uplink is not installed

Selecting the status will route you to the [Uplink Settings](#) screen, where you can view details to help troubleshoot and correct the issue. You can also check these settings to see the most recent time Uplink communicated with Tricefy by reviewing the Uplink Details. The Heartbeat Time should not be more than one minute.

Uplink is not Installed

If you recently purchased a new computer or server, you probably have to install Uplink again

 [Instructions for Setting up a new Computer](#)

Uplink is not Online

Make sure that the computer is turned on, configured so that the drives do not fall asleep, and Uplink is installed.

Next, refer to the following sequenced steps:

1. Confirm the services `trice_boxComm` and `trice_storescp` are running in the background using the Window Services Panel
 - To open the Window Services panel, enter "`services`" into the Start/Search box and select the best match
 - [See other methods for opening the Window Services panel](#)
2. Reboot the Uplink computer/server
 - Check to see if the Uplink is back online
3. [Reinstall Uplink](#)
4. Confirm that the ultrasound system is connected to the internet
 - If the ultrasound system is using a wireless network, try using an Ethernet cable to see if a hardwired connection resolves the issue. If it does, the issue is in regards to the wireless connection.
5. Confirm with the IT department that firewalls and/or proxy servers are not blocking Tricefy communication

 [Firewall and Proxy Server Settings](#)



IP Address

The most common scenario that affects Uplink is a dynamic IP address. If Uplink is installed on a computer that has a dynamic IP address, the IP address will change every time the computer is restarted.

Your ultrasound system relies on this IP address to communicate with Uplink. If the IP address is changed, you need to also change the address in the DICOM settings of your ultrasound system. To prevent this, refer to the instructions below to confirm your IP address is set to **Static**.

 [See instructions to make sure my IP address is Static](#)

If your IP address was set to dynamic (DHCP), correct the setting by selecting the radio button that says **Static** and write down the current IP address.

Next, we need update the IP address in your ultrasound system so that it matches the one you just wrote down. These settings are located on the DICOM screen where we initially setup your system. You can refer to our [Configuration Instructions](#) to help find the settings, but instead of adding a new store like we did during setup, we are going to select the existing Tricefy stores (patient and archive), and edit each one.

From here, we can enter and save the new IP address.
