



# Philips EPIQ Configuration

Last Modified on 05/12/2016 1:43 pm EDT

1. Click the **Support** button on the machine to access the Setup screen
2. Select the **DICOM Settings** tab to access the DICOM Preset screen
3. Click **Add**
4. Select **DICOM Storage Server** in the Device Type Box - Enter the following:
  - Alias or Name = Trice Medical
  - AE Title = archive
  - IP Address = x.x.x.x - refer to the computer running Uplink
  - Port = 104 (or 105\*)
5. Select **DICOM Storage Server** in the Device Type box - Enter the following:
  - Alias or Name = Trice Patient
  - AE Title = patient
  - IP Address = x.x.x.x - refer to the computer running Uplink
  - Port = 104 (or 105\*)

\*[Click here for instructions on determining the correct port.](#)

[Click here for instructions on testing the configuration of your machine.](#)

**Note:** *The machine cannot be in Exam Mode during configuration.*