


Uplink Port & IP Address

Last Modified on 02/25/2018 10:20 am EST


Every computer connected to the internet has a unique IP Address, similar to how every building in your town has a unique street address.

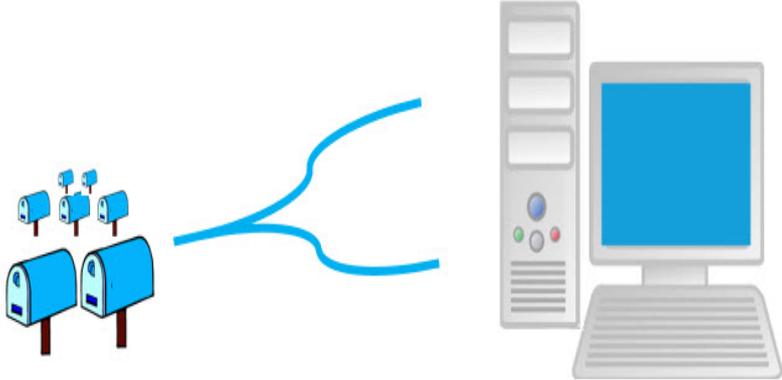

The diagram shows five blue mailboxes on the left, connected by a blue line that splits into two paths leading to a computer system on the right. The computer system consists of a tower PC, a monitor, and a keyboard. Below the computer system is the IP address 01.01.01.001.

Your computer is constantly receiving data. This data is collected using Ports. Consider this like a shopping plaza - the entire plaza has the same address, but each shop receives inventory at their own suite.

When we configure our ultrasound system, the first thing we do is enter the IP address and port number of the computer running Uplink. This information tells the ultrasound system where to send exam data.

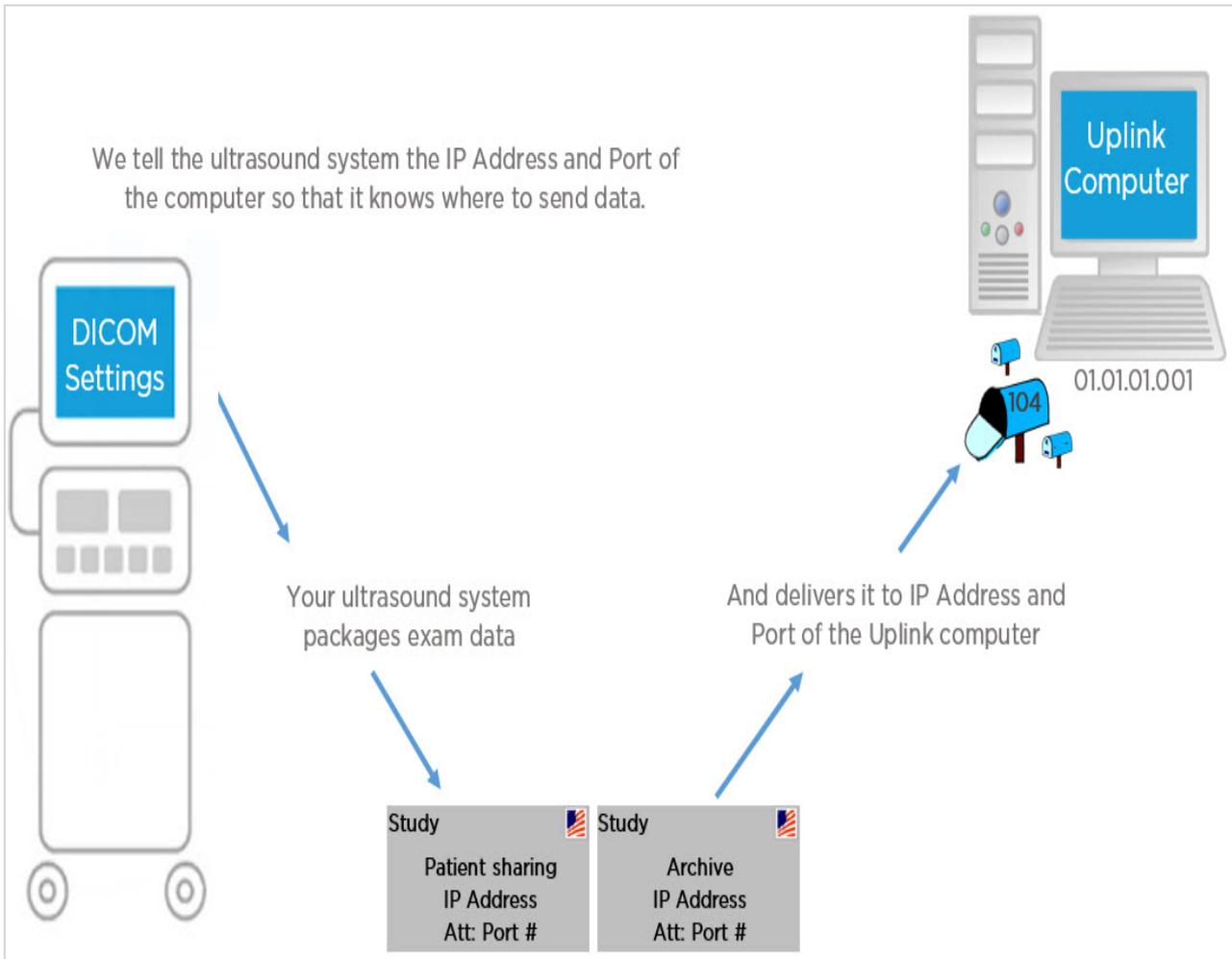


Port Number and IP Address

The port is determined when the Uplink software is installed.

- If the computer is using DICOM software, such as ViewPoint, the port is most likely be 105.
- If the computer is **not** using DICOM software, the port is most likely 104.

After Uplink was installed, the IP address and port number are displayed in a dialog box:

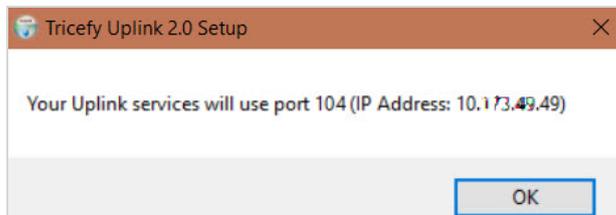

This information is also displayed in Step 2 of the Getting Started screen:

✓
Install Uplink

2
Configure Ultrasound

3
Upload Examination

4
Configure your account

☰
You are ready to Tricefy

Configure Ultrasound System ← →

To configure your ultrasound system(s), enter the IP address of the computer running Uplink and the installation port as a DICOM store. This tells your system where to send data.


Your available Tricefy Uplink Installations are:

Name	Display Name	Account	IP	Port
A1A1AA1AA1A1A1A10000000000	Name chosen at Activation	Name of Clinic Account	↓ 01.11.001.01	↓ 105

Account Settings

If you are setting up additional machines or new machines, the Account Administrator can also find the port and IP address within the Account Settings:

- 1) Select the User Menu at the top of the screen and choose **Accounts**:

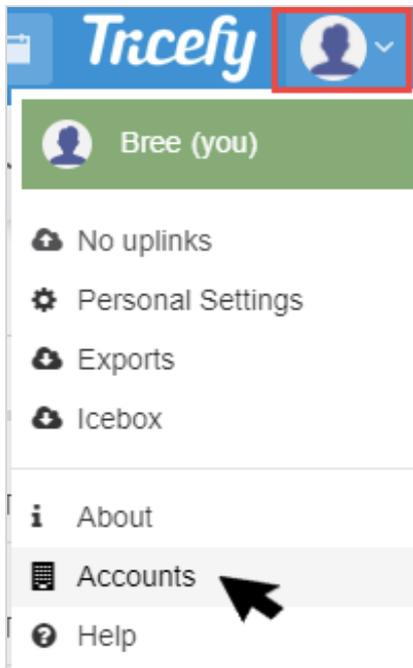

- 2) Select the appropriate account shown under **Account Name**:



3) Uplinks are listed with their respective IP address and port number. If there are multiple Uplinks, you may have to try each IP address to determine which one to use for your system.

Uplinks			
Display Name	IP	Port	Status
macbook	000.000.000.000.	104	ok

Download Uplink

To download Uplink on a different computer, or to reinstall for troubleshooting purposes, select the appropriate icon under **Download Uplink**:

- Select the **Windows** icon if using a Windows computer or laptop
- Select the **Apple** icon if using an iPad, iPhone, or Apple computer
- Select the **Android** option if using an Android phone or tablet