



Philips IU22 Configuration

Last Modified on 02/12/2019 4:11 pm EST

1. Press the **Setup** key on the keyboard
2. Click **Print/Network**
3. Click **Global Configuration** tab
4. Within **Devices**, **Add a New Device**
5. Select **DICOM Archive Server** in the Step 1 drop down

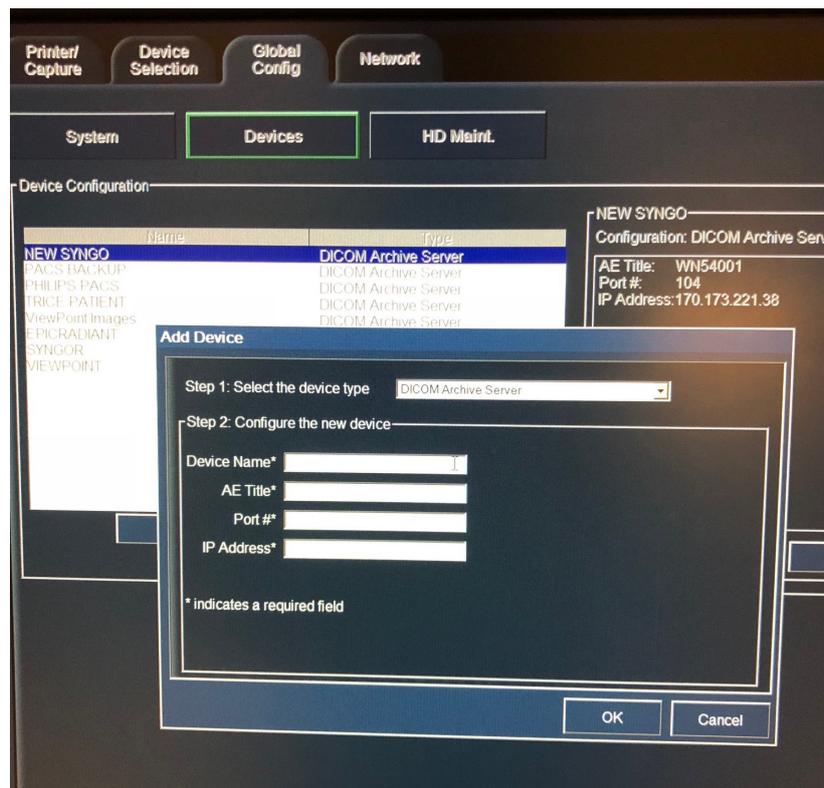

7. Configure the first new device (long term medical image store) as:
 - Alias or Name = Trice Archive
 - AE Title = archive
 - Port = 104 (or what was input during uplink setup)
 - IP Address = x.x.x.x - refer to the computer running Uplink
8. Configure the next new device (texting/email patients) as:
 - Alias or Name = Trice Patient
 - AE Title = archive
 - Port = 104 (or what was input during uplink setup)
 - IP Address = x.x.x.x - refer to the computer running Uplink
9. Activate the New Device by clicking **Device Selection**
10. Click **Archive**
11. Check the active **Archive Servers** for Trice

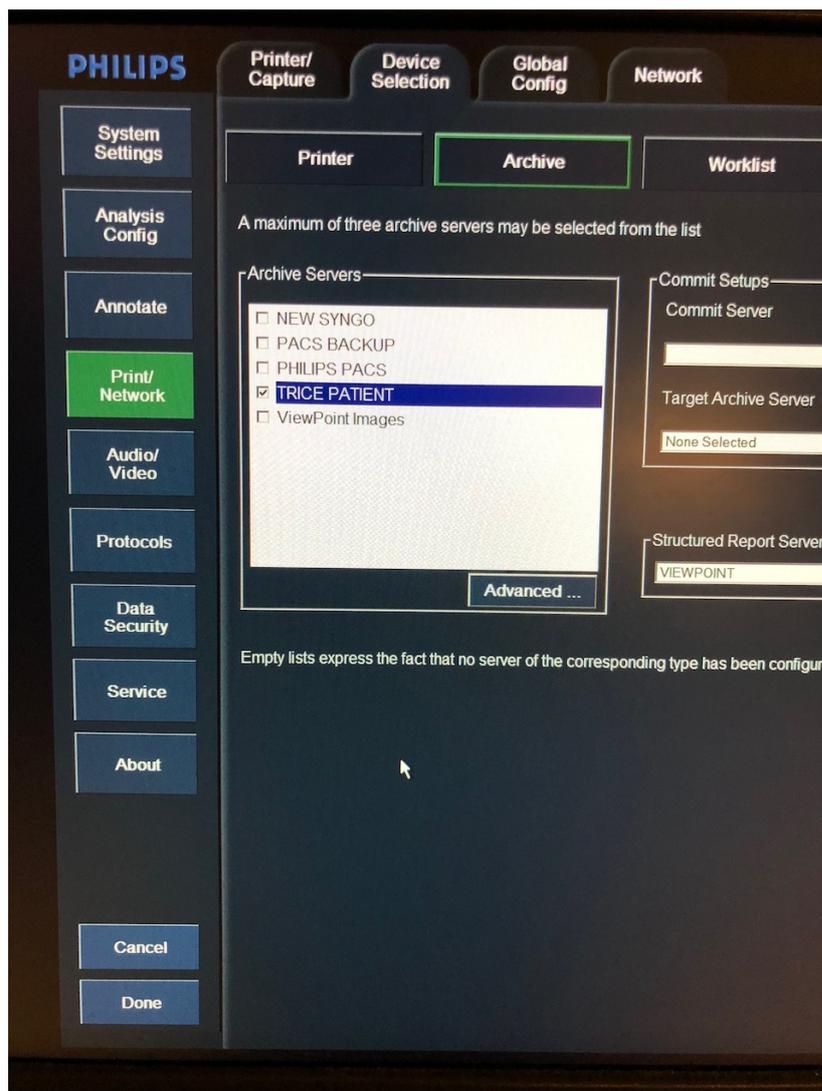

12. Save and Done

[*Click here for instructions on determining the correct port.](#)

[Click here for instructions on testing the configuration of your machine.](#)

Note: The machine cannot be in Exam Mode during configuration.
