



Philips HD11 & HD15 Configuration

Last Modified on 05/04/2017 7:05 pm EDT

Add DICOM configuration to create two new Storage/Archive services:

- a. DICOM Preset screen: Click **Preset rename, Create, Delete...**
- b. DICOM Preset Controls screen:
 - i. Type **TRICE MEDICAL** in the Create a New Preset box
 - ii. Click **Create New**
 - iii. Click **Close**
- c. DICOM Setup screen:
 - i. Confirm **TRICE MEDICAL** appears in the Change DICOM Preset box. If not, choose this option from the dropdown menu.
 - ii. Click **Change Settings for Current Preset**
 - iii. Click **Servers and Roles**
- d. Servers and Roles screen:
 - i. Under Servers, click **Modify**
 1. Type **TRICE MEDICAL** in the AE Title box
 2. Type 104 or 105 in the Port box ([Click here for more information](#))
 3. Select IP Address under Host/IP Address
 4. Type the IP Address **x.x.x.x-archive** - refer to the computer running Uplink for the IP address
 5. Click **Done**
 - ii. Under Roles, click **Modify**
 1. Under *Storage > Primary Storage SCP*, select **TRICE MEDICAL** in the drop-down menu
 2. Click **Advanced**
 3. On the Network Preferences tab:
 - Select the **Image Format** tab
 - Under *Color Images>Format*, select **RGB**
 - Under B&W Images, select **Use MONOCHROME2 Format**
 - Under Transfer Syntax, select **Uncompressed, Implicit VR Little Endian**
 - Click **OK**
 4. Select **Done** and **Close** under the Roles section
 5. Click **Modify Current** when the DICOM Preset Settings Changed dialog box appears, followed by **OK**

Click **Close**, followed by **OK** to close the Setup screen. Repeat the above steps to create the second archive:



- In Step B, type TRICE PATIENT instead of TRICE MEDICAL
- In Step D, replace the “archive” in the IP address with “**patient**”

[Click here for instructions on testing the configuration of your machine.](#)

***Note:** The machine cannot be in Exam Mode during configuration.*
