

Studies without DICOM

Last Modified on 05/23/2019 4:51 pm EDT

When studies are created with your imaging system, **DICOM** information (the stuff you type into your imaging machine before or during an examination) is sent to Tricefy. This includes the patient's name, date of birth, etc.

If you are creating a study without your imaging system, such as a study that only contains reports, you'll need to enter that information (or choose the patient) manually.

First, create a study by selecting **Create Study** from the **+** menu on either the Studies or Consults screens:

The screenshot shows the 'Studies' interface. At the top right, there is a red-bordered dropdown menu with a '+' icon. The menu is open, showing two options: 'Create Study' and 'Upload DICOM'. An arrow points to the 'Create Study' option. Below the menu is a table of study entries:

	Full Name	Study Date	Status	Study Details	Interface Status
TE	Anonymous	2018-02-21 4:14 PM MST	None	1	Fail
TE	Anonymous	2018-02-21 1:05 PM MST	None	2 1	Success

On the right side, there is a sidebar with 'ACCOUNT ACTIVITY' and a user profile for 'Josh' with a 'Viewed study: "2018-02-21 - ANONYMOUS"' entry.

A pop-up window will display for entering the name of the patient and the date of the study:

The 'Create a new study' form has the following sections:

- Account:** A dropdown menu with 'Delicious Waffles' selected. Below it, the text reads: 'This is the account associated with the study.'
- Patient:** An empty text input field. Below it, the text reads: 'This is the patient associated with the study.'
- Study Date & Time:** A text input field with '02/21/2018 5:19 PM' entered. Below it, the text reads: 'This is the date and time that the study was initially recorded.'

At the bottom left, there is a 'Create Study' button.

Tricefy will locate the patient before you finish typing the name and auto-populate possible options in a drop-down menu for you to select:

The patient selection dropdown menu shows the text 'geral' entered in the input field. Below the input field, a list of suggestions is displayed, with 'Dean Gerald' highlighted in a green bar.

If the patient has never had a study in Tricefy before, you'll be prompted to add them:

Patient

Gerland

"Gerland " was not found.

Create a new patient?

- Select Create New Patient
- A pop-up window will display so that you can enter their first name, last name and date of birth

Next, select inside the **Study Date & Time** field to display a calendar for choosing the date of the examination. Select the [blue clock](#) icon for specifying a time:

Calendar for March 2017:

Su	Mo	Tu	We	Th	Fr	Sa
26	27	28	1	2	3	4
5	6	7	8	9	10	11
12	13	14	15	16	17	18
19	20	21	22	23	24	25
26	27	28	29	30	31	1
2	3	4	5	6	7	8

03/16/2017 3:05 PM

Lastly, select the **Create Study** button to create the study.

Once the study is created, it will automatically open in the viewer. Select the **Choose Files** button (shown below in the [red](#) box) to attach a report, PDF document, or image file to the study.

Patients Studies Consults

To upload study images, please use the Attach Files functionality in the sidebar.

Search...

Tricefy

Dean Gerald
2018-02-21 5:21 PM MST

Send to... + -

Dean Gerald
2018-02-21 5:21 PM MST

INFO NOTES SHARES ACTIVITY

Study Date
2018-02-21 5:21 PM MST

Study Expires
2043-02-22 5:22 PM MST

Status
None

Attach File(s)
Choose Files No file chosen

Labels
+