


Attaching Non-DICOM Files to a Study

Last Modified on 2021-03-04 15:32

Non-DICOM files, such as PDF documents, can be attached to studies.

First, select the study on the Study List or open the study in the viewer.

Next, select **Choose Files** from the side Info Panel. If this panel does not appear, select the  icon on the right side of the screen.

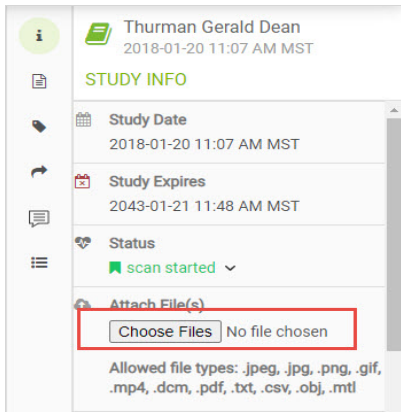

A window will display for choosing files from your computer or network. These can include JPEG, MP4 and any of the other file types listed on the screen.

Please note that DICOM images cannot be attached to existing studies since they have a different study unique ID. Refer to the link below for instruction on adding DICOM to a study already in Tricefy.

 [Instructions on adding DICOM files to an existing study](#)
