


Send to Doctor

Last Modified on 2024-03-13 14:10


Sharing a study with another physician allows them to view the study, provide input, answer questions, and prevents patients from logging around printouts of their medical images.

Each time a study is sent to a doctor, it creates a **Consultation**. These are listed on the [Consults screen](#).

To send a study to a doctor, select the study (or specify thumbnails) and choose **Send to Doctor** from the Send To menu. This menu is available on the [Studies screen](#) and [Viewer](#) (and is only visible if a study is selected):

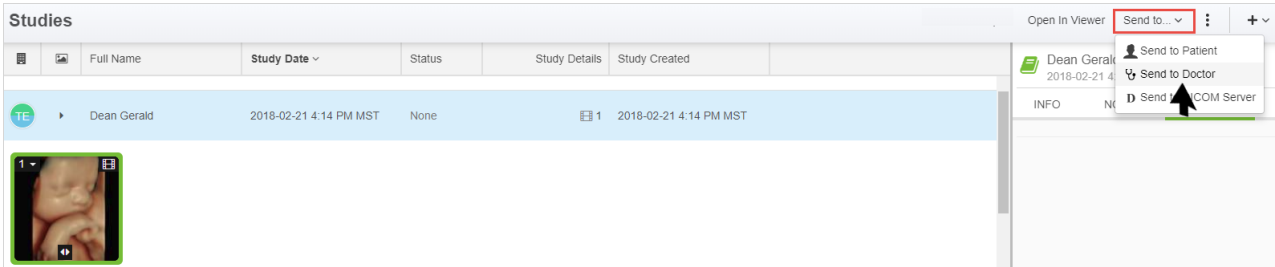

A pop-up window displays for entering the doctor's contact information:

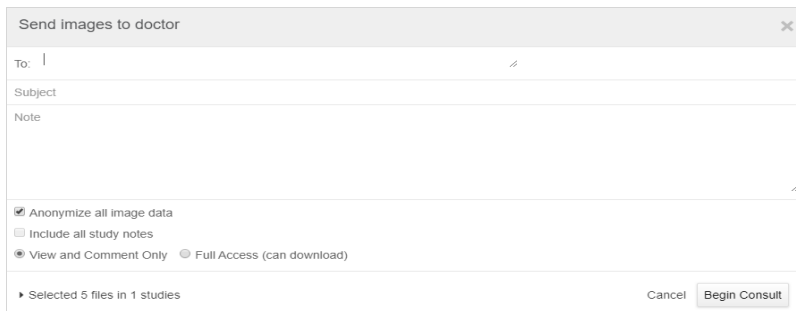A screenshot of a pop-up window titled 'Send images to doctor'. It contains the following fields and options: a 'To:' field for the doctor's email address, a 'Subject' field, a 'Note' field, a checkbox for 'Anonymize all image data' (checked), a checkbox for 'Include all study notes' (unchecked), and radio buttons for 'View and Comment Only' (selected) and 'Full Access (can download)'. At the bottom, it shows 'Selected 5 files in 1 studies' and two buttons: 'Cancel' and 'Begin Consult'.

- Provide the **Email Address** of the doctor or clinic
- Create a **Subject**
- Enter a **Note** (optional)
- Check **Anonymize all Image Data** to remove identifiable patient information from the shared version
 - This option cannot be applied to reports, pdf documents and other non-DICOM files
- Check **Include all Study Notes** to allow the invitee to view existing notes regarding the study
- Choose permissions for the recipient:
 - **View and Comment Only**: The recipient will have the ability to view the examination and leave notes
 - **Full Access**: The recipient will have the ability to download and modify the examination

Select **Begin Consult** to share the study. Recipients receive an invitation to join the consult via email:

 [See an example email that is sent to doctors when sharing a study](#)

Once the invitation is sent, the study appears on the [Consults screen](#). From this screen, the study can be sent to additional colleagues as well.


To see study images that were sent, select the study on your [Studies screen](#), followed by the **Shares** tab of the side panel.

INFO NOTES **SHARES** ACTIVITY



PATIENT LINKS

Delivered to **abh@imaging.com** 2 months ago ▾

CONSULTS

 **nic@imaging.com**

Sent 1 files 2 months ago

-  Nick Doc
Unread
-  nick+doc2@aging.com ⓘ
Unread

***Note:** Doctors do not need to be members of Tricefy in order to receive and participate in consults. They will be prompted to create a password and only have access to the study that was shared.*
