

Printing Images and Reports/PDFs

Last Modified on 2020-10-26 11:55

To print images, documents and reports, first download the file from the Study List by selecting the drop-down in the upper-left corner of the thumbnail:



- To download an image file, select `jpg`
- To download a PDF file, select `application/PDF`

Open the downloaded image and print as normal.

Printing PDF Documents/Reports

You can also print PDF documents if they are opened in the viewer. To do so, hover the mouse over the document and select the printer icon:

The screenshot shows a PDF report viewer displaying a 'FIRST TRIMESTER ULTRASOUND REPORT'. The report content includes patient information, a greeting, and several data tables. A printer icon is highlighted in a red box in the top right corner of the viewer interface.

Hospital No: []
Exam Date: 9 August 2010

Dear

Thank you for referring your patient

.....

INDICATION:	Nuchal translucency scan. For first trimester combined screening.		
History:			
Maternal age:	31 years		
Last period:	15 May 2010		

EDD by ultrasound:	13 February 2011	EDD by dates:	19 February 2010
Gestational age:	13 weeks + 1 day	Gestational age by dates	12 W+ 2 D*

First Trimester Ultrasound:			
Transabdominal US with Voluson E8	Ultrasound view:	Good	
Fetal heart action present	Frequency	150	bpm
Crown-rump length (CRL)		73.4	mm