

Deleting a Study or Image

Last Modified on 2026-03-12 13:46

Deleting a study or image from Tricefy is permanent -- it cannot be undone.

Deleting Individual Images or Files


To delete individual images, clips, or other files, select the drop-down menu on the top left (next to the capture number) of any thumbnail and select **Delete**:



Deleting an Entire Study

You can also correct the information using Tricefy

To delete a study or subset of selected images, select the study (or studies) on study list and choose **Delete** from action menu (3 vertical dots icon). If only **certain instances within the study are selected**, only those instances will be deleted.

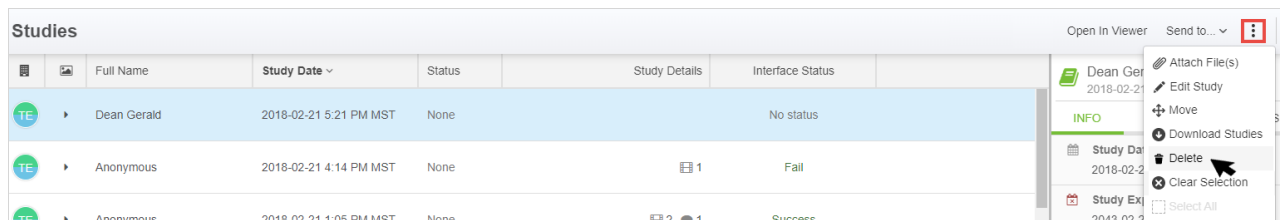

A confirmation popup window will display to make sure you really want to delete the study:



- Click the red **Yes, I want to delete the Studies Listed above** button to delete the studies
- To cancel, select the X in the upper-right corner of the window