


## Step 2: Configure your Voluson

Last Modified on 2020-09-18 15:17

After you activate your Tricefy account, it only takes a few seconds to setup your machine. You can automatically send data to Tricefy at the end of each exam (by pressing the **End Exam** button) and/or share exams with patients by pressing a P button.


First, select the **Button Configuration** tab:

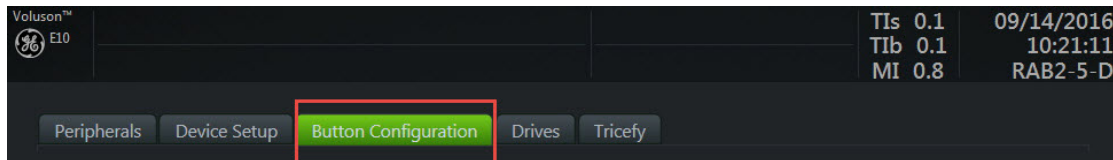

- For archive-only users, refer to the **Archiving** step immediately below
- For patient-sharing only users, refer to the **Patient Sharing** step
- Refer to both steps if using both the archiving and patient sharing features


## Archive: Sending Data at the End of Exam

To send data when the exam has ended, select the **End Exam** option and make sure **Move Clipboard Content to Internal Archive** and **Store Clipboard Content to Tricefy** are selected in the Clipboard Content box.

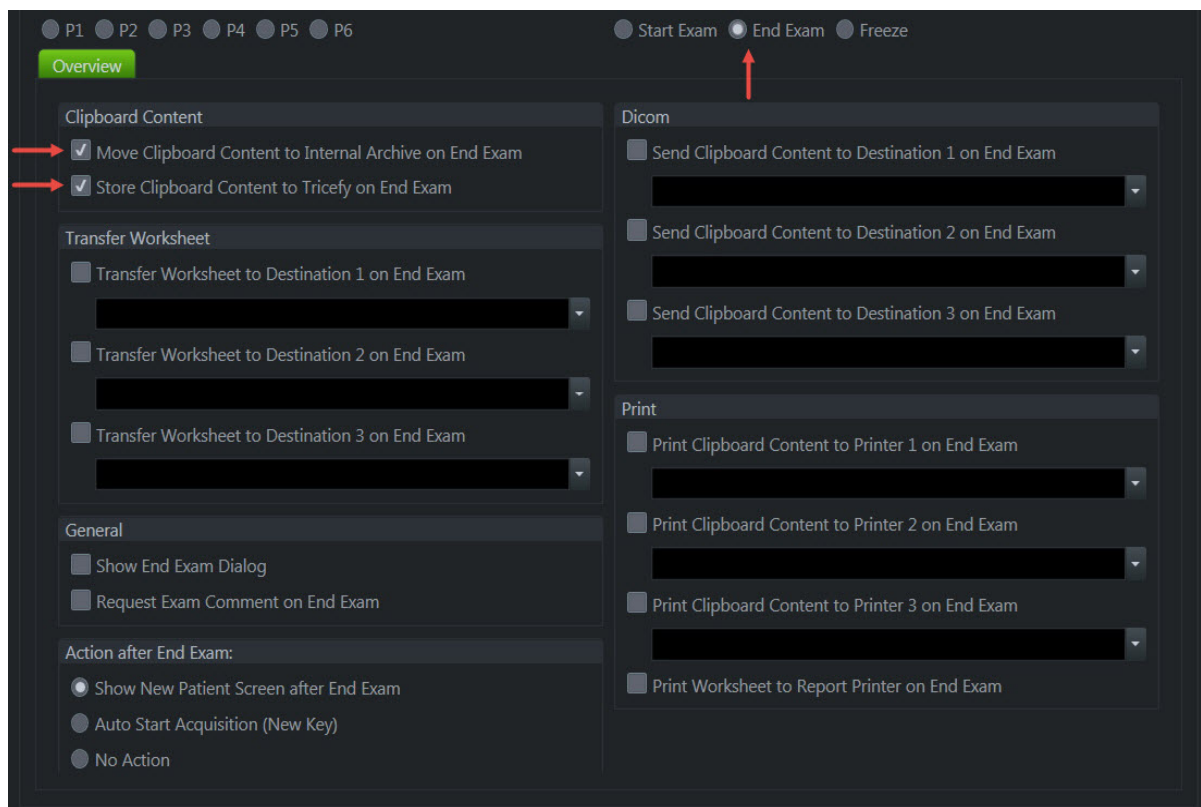

## Patient Sharing: Sending Data with a P Button

To send data by pressing a P button, select the P button you would like to use (P2) and make sure **Save to Clipboard with P2** is selected in the General box. Select **Share Image/Cine with Patient with P2** in the Tricefy box.

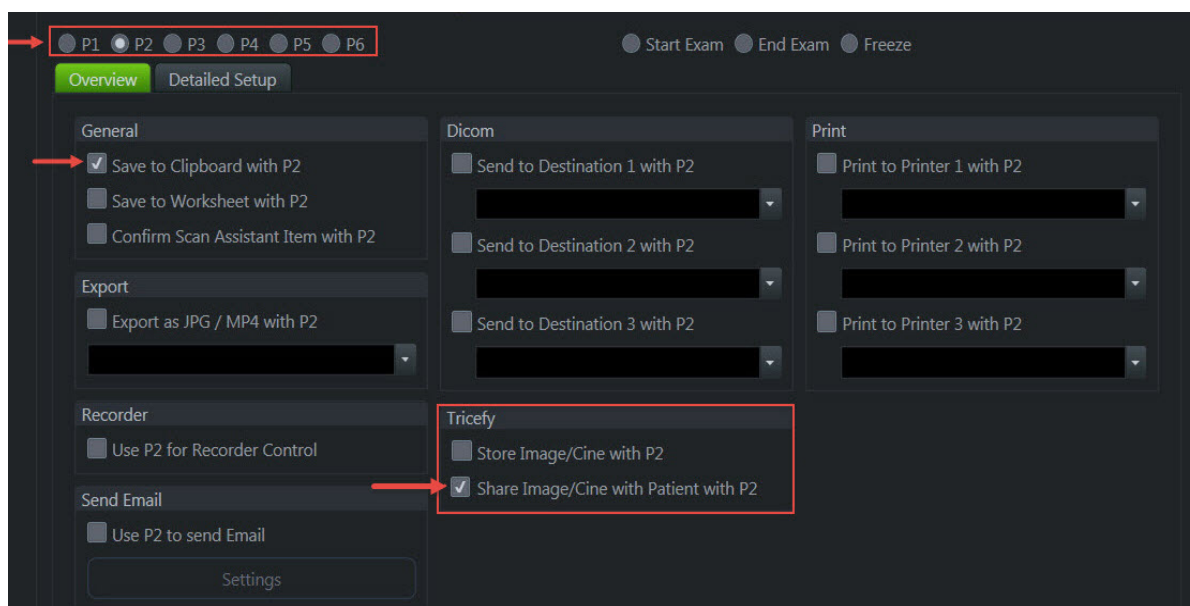

If you have ultrasound systems that do not have Tricefy Inside but would like to connect them to Tricefy account, please refer to our standard [Getting Started Guide](#) for configuration steps.

 [Move to Step 3 - Sending a Study](#)