


Step 1: Install Tricefy and Activate your Vscan


Last Modified on 05/09/2017 10:40 am EDT

The following steps will have your Vscan connected to the Tricefy Cloud in just seconds!

1) Select the menu icon (shown below in the red box) and choose **Settings** from the menu:

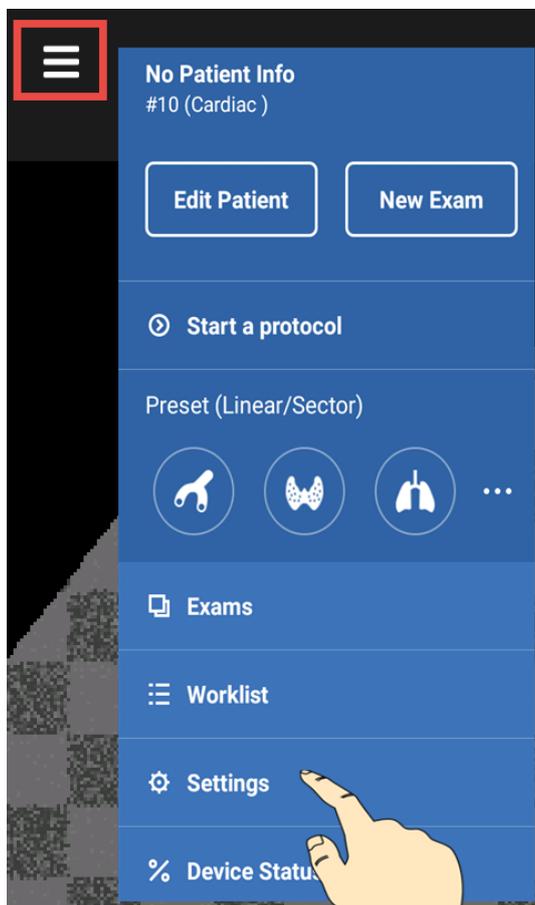

2) Open the **GE Marketplace** (under System Settings) and tap the **Install** button to install your Tricefy software:

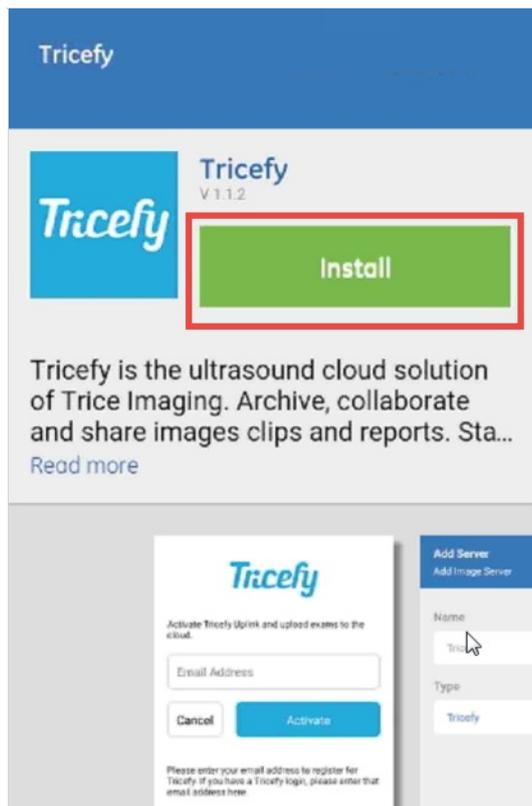

3) Select the **Image Server** option within the settings (under **Server Settings**), and choose **Tricefy** (which will automatically also be the **Name** of the server).

- Tap the **Add** button to add your new Tricefy server

Settings
What would you like to change

Set Current Preset as Default
Cardiac

Server Settings

Worklist Server Verify

Image Server Verify

DICOM Encoding
Select Character Encoding

Add
Add Image Server

Name
Enter a memorable name

Type
Dicom image server
Windows share
Tricefy


Port

Verify

4) A message will display as Tricefy initializes (1); you may not even notice this screen since the process completes in just one second. Enter your email address and tap the **Activate** button (2). Tapping the Email Address text box will display a keyboard.

- If you have a Tricefy account, enter the email address that you use to log into Tricefy

If your Vscan has already been activated, it will skip to the final screen (3).

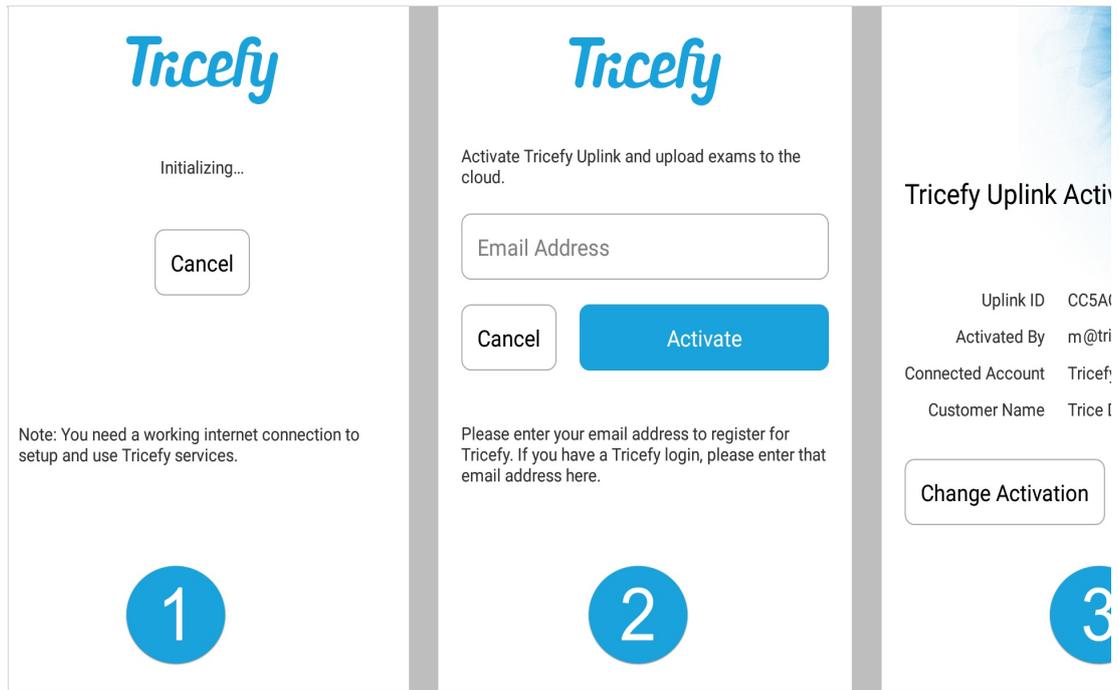


If a different department or a user with a different Tricefy account would like to use the Vscan, simply select **Change Activation** to enter a different email address.

- Email addresses are saved and auto-populated based on the first few letters so that you can easily and quickly switch between users/departments

Otherwise, select **OK**.

5) Tap **Add** to complete this process:

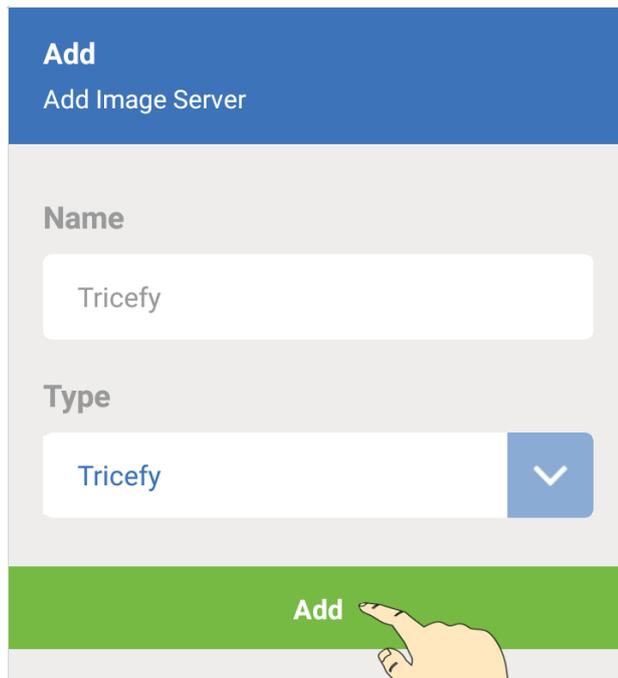

Your Vscan is now connected to the Tricefy Cloud!