


Step 2: Configure your Vscan

Last Modified on 05/09/2017 10:41 am EDT

After you install Tricefy and activate your Vscan, you will see **Tricefy** listed under Server Settings:

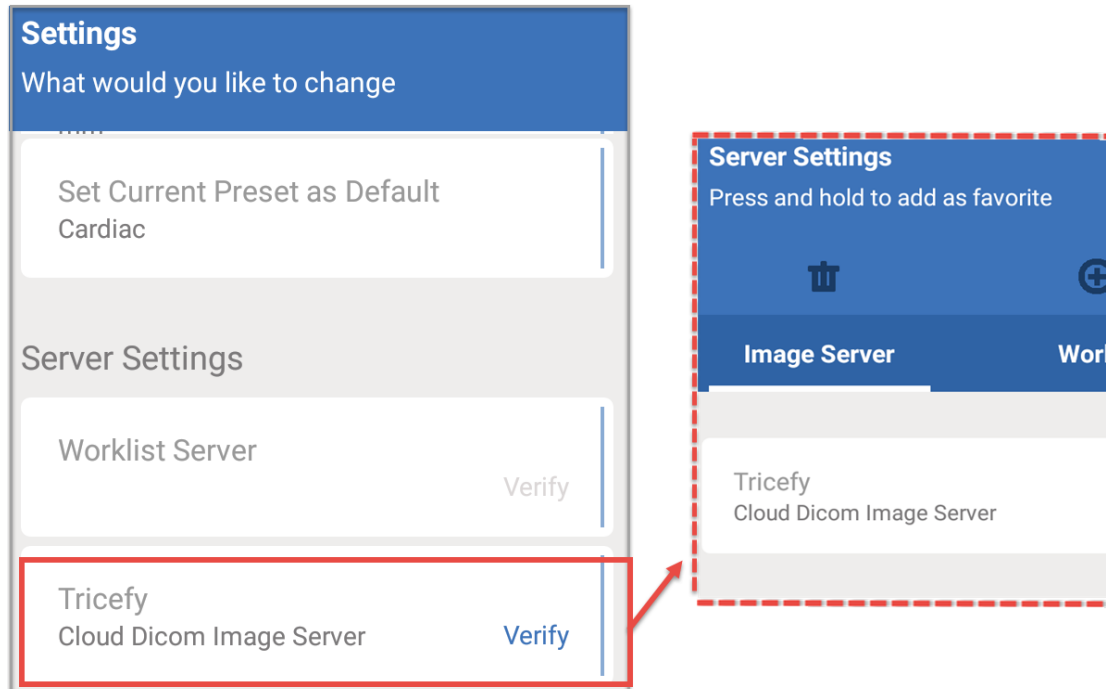


To make sure your Vscan can communicate with your new Tricefy server, tap **Verify** to perform a test.

If everything is working correctly, a **Verify OK** message will display.

- If your test does not work, make sure your Vscan is connected to the internet

Make it your Favorite

Tap and hold the **Tricefy** button (listed under your Server Settings, as shown above) to make it your "favorite" (default) server. Select **Add** and your examinations will automatically upload to Tricefy!

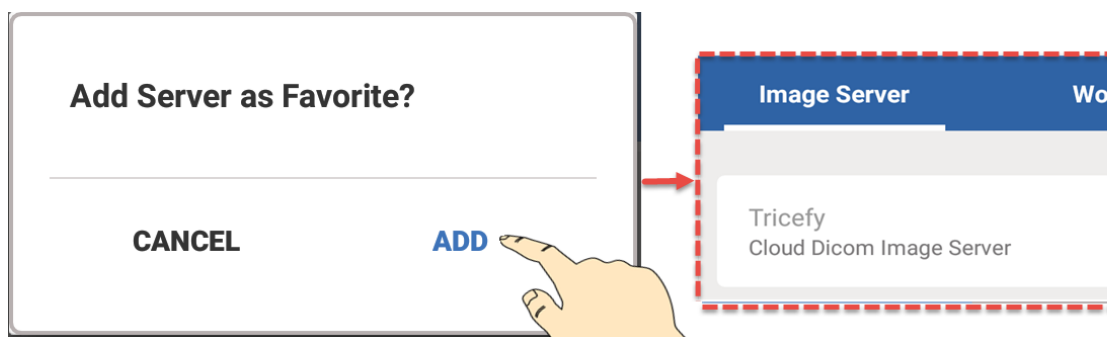

You will know that Tricefy was successfully added as your favorite because it be marked with a star.

Remember, in order to save an examination to any server using the Vscan, patient data for that exam must be entered.