
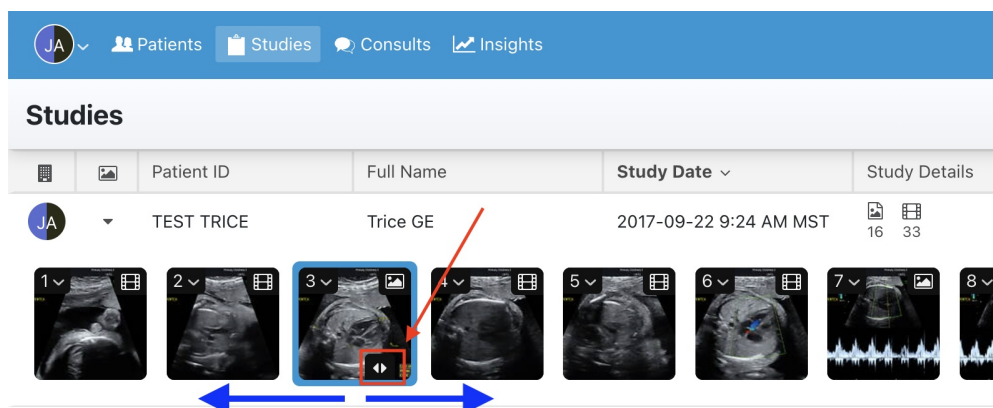


Rearranging thumbnails in an Examination

Last Modified on 2025-07-17 11:02

The order of study images can be rearranged from the study list. Expand a study in the list, click and hold the  icon on a thumbnail, and then move the image left or right. A **blue** box will appear around the thumbnail (shown below) while moving the image - drag the thumbnail left or right within the study.



Repeat the process for any additional thumbnails you would like to move.

The viewer will always display thumbnails in consecutive order; if you want to look at three images side-by-side, arrange them next to each other on your study list.

 [See instructions for changing the layout of your viewer](#)

If you want to compare thumbnails from different studies, select the desired thumbnails in each study and choose **Compare** from the study menu.

 [See instructions for comparing thumbnails from different studies](#)