



Android Device Installation Instructions

Last Modified on 2023-04-06 12:18

Tricefy can be installed on Android devices (version 4.0 or higher).

Due to variances in manufacturers and models, instructions may slightly vary.

Tricefy will install as a .apk file, which is the file type used when installing Android applications.

Step 1: Install App from the Google Play Store

 <https://play.google.com/store/apps/details?id=com.tricefy.uplink>

Once the application is installed, a Tricefy icon will be located with your applications.

Step 2: Log into Tricefy and Activate your Device


You will be prompted to enter your username and password (that you use to log into Tricefy) when you initially open the application. This will link your device to your Tricefy account. Next, you will be prompted to log into Tricefy (if you are not currently logged in on your Android browser).

Once you are logged in, select the blue **Activate Tricefy** button.

Step 3: IP address and Port Number

When you open the Uplink application (by selecting the icon on your device), you will see the IP Address and Port number of your Uplink.

Very often the Tricefy Uplink app is on the same device as the imaging device app, so you will want to enter in the loopback IP address into the app's DICOM send/export configuration **127.0.0.1**

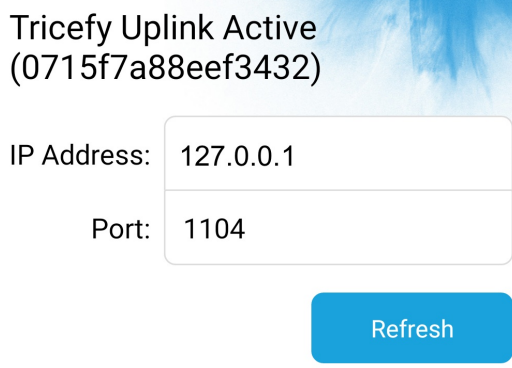

Tricefy Uplink Active
(0715f7a88eef3432)

IP Address: 127.0.0.1

Port: 1104

Refresh

Learn more about setting up your imaging machine by visiting our [Configuration Help Page](#).
