


Philips Lumify

Last Modified on 2021-01-04 13:01


To setup your Philips Lumify ultrasound system, first install the Tricefy Uplink application on your Android phone.

See instructions for installing the Tricefy App

Next, open the Lumify application and select **Export Destinations** from the settings menu:

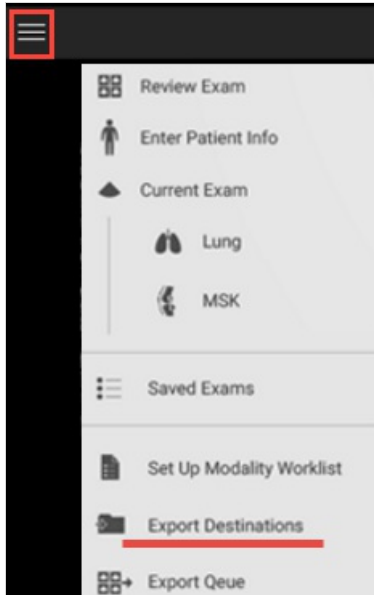

Select **Add New Destination** - we will repeat this process to add two destinations: one for sharing with patients and one for archiving:

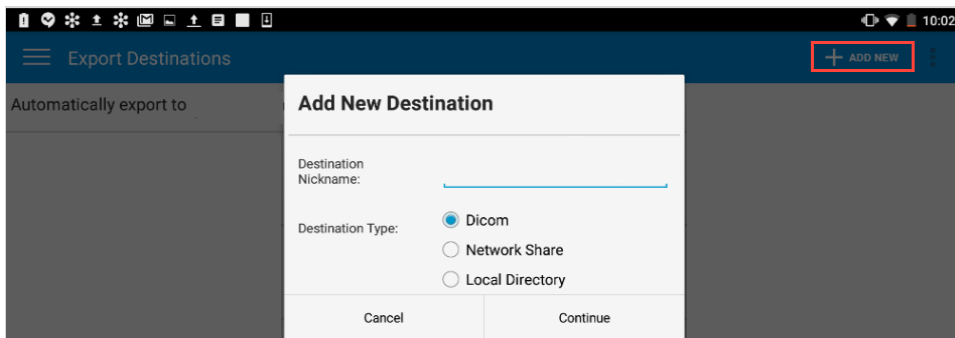

For the first destination, enter "Patient" as the Nickname, DICOM as the Destination Type and select **Continue**:

Trice Patient

Lumify AE Title:

Remote AE Title:

Hostname or IP:

Port:

Export Format

RLE (Lossless) JPEG (lossy)

Show Advanced Options

Cancel Test Save

Enter the following information:

- Lumify AE Title: Trice Patient
- Remote AE Title: patient
- Hostname or IP: Refer to the Tricefy Uplink application on your phone **
 - If using the Tricefy Uplink app, enter 127.0.0.1
- Port: Refer to the Tricefy Uplink application on your phone **

** Selecting the Tricefy icon on your phone will open a window that contains the IP address and Port number:

Tricefy Uplink Active
(0715f7a88eef3432)

IP Address:

Port:

Refresh

To make sure everything works correctly, tap the **Test** button; a **Success** message will appear. If this message doesn't appear, make sure the IP address and Port number are entered correctly and that your phone is connected to the internet.

Select **Save** and repeat the above process to add your second destination with the following information:


- Lumify AE Title: Archive
- Remote AE Title: archive
- Hostname or IP: Refer to the Tricefy Uplink application on your phone
 - If using the Tricefy Uplink app, enter 127.0.0.1
- Port: Refer to the Tricefy Uplink application on your phone

How it Works

After you complete an examination using the Lumify ultrasound system, select the Settings menu (☰) and choose either **Current Exam** (to send the current examination to Tricefy) or **Saved Exams** (to send previous exams to Tricefy).



To send one exam, select the exam and choose an option from the **Export Exam** menu in the upper-right corner. The two new locations we created will be listed:

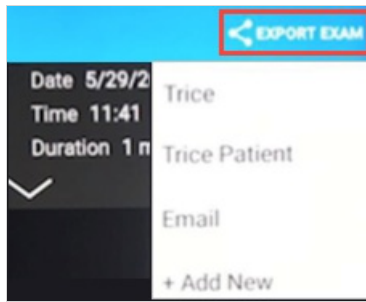

To send multiple exams, tap and hold one exam from the **Saved Exams** list. After a few seconds, you will be able to select additional exams. Select an option from the Export Exam menu to send all the selected exams to Tricefy.

A message will display once the exam was exported to Tricefy.
