


## Send To Menu

Last Modified on 2021-03-18 11:10

The **Send To** menu is located at the top of your viewer and Studies screen (if a study is selected):

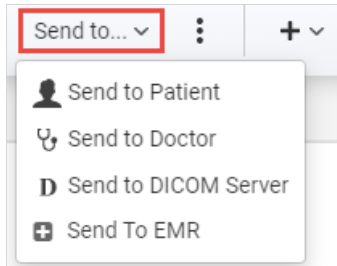

- **Send to Patient:** Sends an examination to a patient. They will receive a link to their own special webpage hosting their images.

 [Learn more about sending to patients](#)

- **Send to Doctor:** Sends an examination to a doctor, such as a referring physician. They will receive a link to view and comment on the exam.

 [Learn more about sending to doctors](#)

- **Send to DICOM Server:** Sends an examination to an external server, such a PACS. Tricefy integrates with outside software and servers to accommodate efficient workflows.

 [Learn more about sending to external servers](#)

- **Send to EMR:** This option is only available for users integration. A pop-up window will display for sending an exam link to your EMR.

 [Learn more about sending to an EMR](#)

This menu is also located on the Consults screen but only **Send to DICOM Server** is available.

- To send the exam to patient, return to your Studies screen by selecting **Studies** from the navigational options:



- To send the exam to additional doctors, use the side panel on the Consults screen

 [Learn how to invite more participants using the Consults sidebar](#)

---