




# LOGIQ E10 with Tricefy Inside

Last Modified on 2024-05-28 18:07

The GE LOGIQ E10 ultrasound systems have Tricefy pre-installed. Additional software is not required in order to use Tricefy, you can your account once the device has an internet connection without a firewall or anti-virus using the following steps:

*\*You must be logged in as an ADM (admin) to make any changes on the Logiq.*

Press the **Utilities** button on the front panel (this is a physical button), and select **Connectivity** using the tabs, followed by Tricefy:

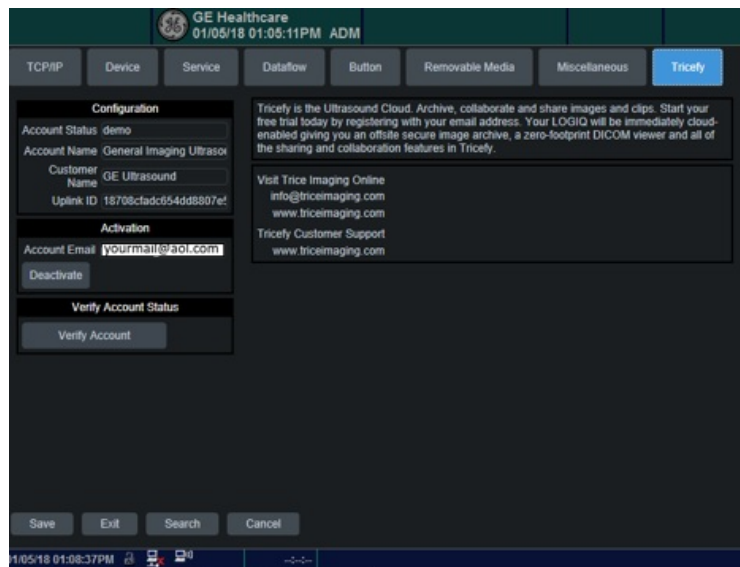


- a) Enter your email address to authorize the account (this should only be done by an IT or clinic administrator)
- b) Select **Activate Account**. This process may take up to 20 seconds.

## Sending Images to Tricefy

You can program your system to either automatically send exam data to Tricefy once the exam has ended or when a P button has been selected. Both options are described below.

### Sending Data with a P Button

To send data by pressing a P button, select the **Button** tab:

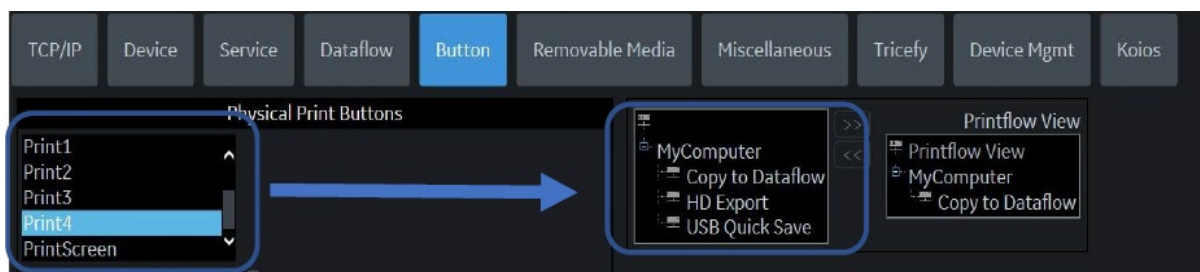

Select the button you would like to use in the left blue box (P1, P2, P3, or P4) and choose a destination from the right blue box. When you have Tricefy activated, there will be additional options:

- Select **Tricefy\_store** for archiving




- Select Tricefy\_Patientshare for patient sharing

If you are using both archiving and patient sharing, repeat this process and choose a different P button.

## Sending Data at the End of Exam

To send data when the exam has ended, select the **Dataflow** tab. When using the End of Exam option, the entire option will be archived (or sent to the patient). To archive or send individually images, use a P button instead.

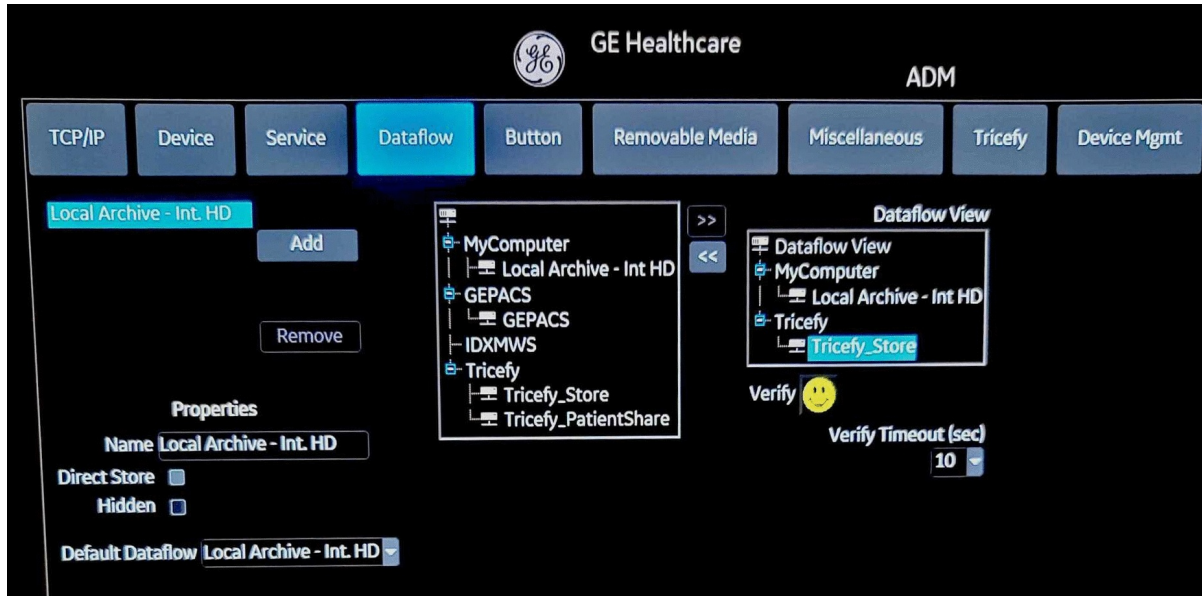

To configure the End Exam button to archive the entire study, move the Tricefy\_store into the Dataflow View. To configure the button to send the entire study to a patient, move the Tricefy\_Patientshare.

## Sending Worksheets and Reports

Refer to [tricefy.help/help/pdfsendvol](http://tricefy.help/help/pdfsendvol) to learn how to send worksheets and reports to Tricefy in one easy step.

## Patient Sharing

When patient sharing from an LOGIQ E10, there is not a field for entering a phone number or email address for the patient. Instead, you can put those entries into the **Patient Comments** or **Visit Comments** (both under Details).

When setting up your automation rule, make sure to add these fields so that Tricefy knows where to find patient contact information:

<b>DICOM Field Name</b> <input type="text" value="PatientComments"/> <span style="color: red;">✘</span>
<input checked="" type="checkbox"/> <b>Find Emails</b> <input checked="" type="checkbox"/> <b>Find Phones</b> We will look in this field for something that looks like an email, phone number, or account.
<b>DICOM Field Name</b> <input type="text" value="VisitComments"/> <span style="color: red;">✘</span>
<input checked="" type="checkbox"/> <b>Find Emails</b> <input checked="" type="checkbox"/> <b>Find Phones</b> We will look in this field for something that looks like an email, phone number, or account.



 [Learn more about automation rules](#)

---