




## Worklist (athenahealth)

Last Modified on 2021-01-08 09:10

You can use your ultrasound system to query athenahealth for patient information using worklist.

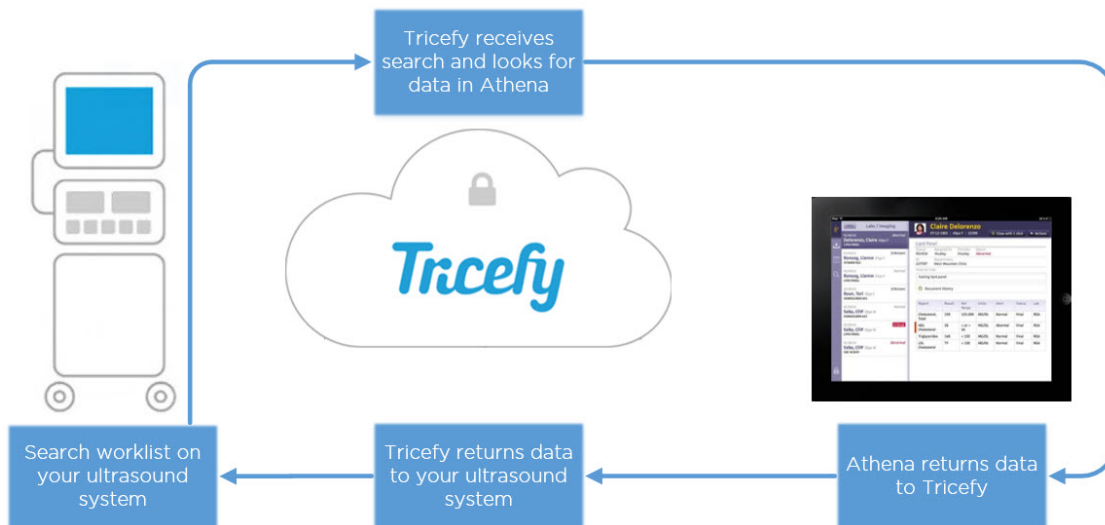

To setup worklist, add a new DICOM Service:

1. Locate the DICOM Configuration screen (sometimes also called the "Device" or "Device Setup" screen)
2. Select the **Add** or **New** button and choose Worklist from the Services drop-down menu
3. Enter the following information:
  - AE Title = Worklist
  - IP Address = Enter the IP address of the Uplink computer \*
  - Port = Enter the installation port (often 104 or 105) \*

\*\*You can view the IP address and installation port within your Uplink Account Settings

4. Select **Save**

Your worklist is active once the athena automation rule is configured.

 [Learn how to setup athena automation](#)

## Worklist Results

The following data will auto-populate into your ultrasound system, as long as the data was included in the athena order. This will prevent needing to manually enter information:

- Patient Name
- Patient ID
- Patient Birth date
- Patient Sex
- Patient Address
- Patient Phone
- Referring Physician
- Performing Physician Name
- Scheduled Performing Physician Name



- Study Description / Exam Time
  - Scheduled Procedure Start Date
  - Scheduled Procedure Start Time
-