


# Downloading an Exam/Study

Last Modified on 2024-04-17 15:52

## Download a Study onto a CD / DVD / USB

To export data from Tricefy (for purposes such as embedding video into a PowerPoint, submitting exams for accreditation, or sharing images for educational engagement), you can download full exams or individual images using the **Download Studies** feature.

Content will download on your local computer as a packaged .zip file.

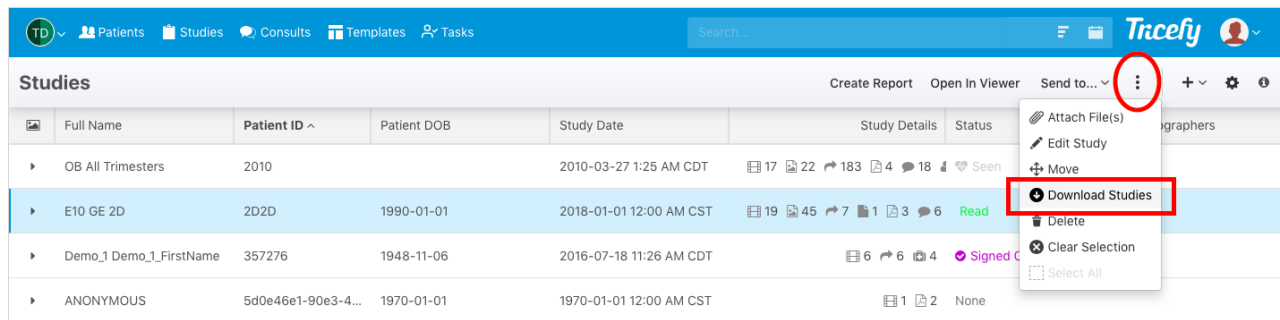

1. On your Study List, highlight the entire study or select individual images/cines/reports by opening the thumbnails
2. Click the three small dots (ellipse button) in the top right corner and select **Download Studies**
3. A pop-up window will display; choose the appropriate options:

Download Studies

Download Format

DICOM
PDF
JPG/MP4

Generate DICOMDIR file

No, I do not need a directory file for DICOM file sets
Yes, I need a directory file for DICOM file sets

Anonymize Data

No, download original study images
Yes, remove identifiable information

Selected 3 files in 1 studies
Cancel
Begin Download

- **Download Format:** Select the file type for the downloaded content
- **Generate DICOMDIR File:** This is only available if the downloaded content is DICOM
  - Select **Yes** to create a DICOM directory for your exam
- **Anonymize Data:** Choose to remove identifiable information (such as patient name and birthdate) or to download the exam with all the data
  - Please note that only DICOM data can be anonymized


Select **Begin Download** to export the data from Tricefy. A message will display while this process takes place.

## Exports Screen

After selecting Begin Download, you will automatically be redirected to the Exports Screen within [Tricefy4.com](https://Tricefy4.com). You can also access



this screen (to retrieve downloads at a later time) by selecting **Exports** from the user menu:

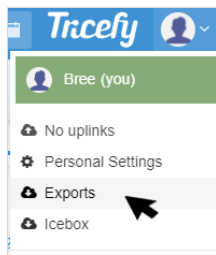


When your exam is ready, you will see a **Download (.zip)** link listed on this screen. You may need to wait a few seconds for the study to compress into a .zip folder. If the link does not appear within a few seconds, select the **Refresh** button in the top right corner to push the process. Larger exams may take longer.

Once complete, select the **Download (.zip)** link to download the contents onto your local computer.

Search...				
Tricefy				
Refresh				
Files	Format	Created Date & Time	Download	DICOMDIR included?
12	dcm	2019-05-22 12:30 PM CDT	<a href="#">Download (.zip)</a>	No
12	web	2019-05-22 12:29 PM CDT	<a href="#">Download (.zip)</a>	No
1	dcm	2019-04-24 3:00 PM CDT	<a href="#">Download (.zip)</a>	No
16	pdf	2019-04-05 3:10 PM CDT	Not Ready	No
16	pdf	2019-04-02 8:24 AM CDT	Not Ready	No

After you select the link, your content will be in a .zip folder (typically in your computer's "Downloads" folder):

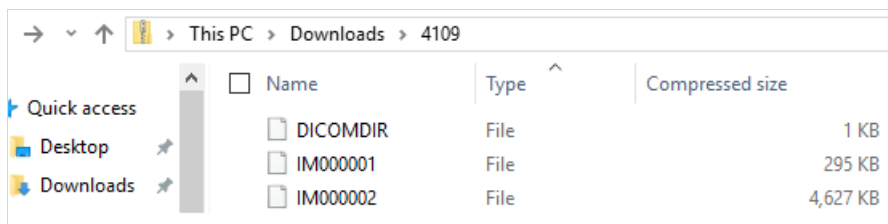

From this location, you can unpack the .zip file and then burn the exam files to a CD, copy it onto a USB drive, or upload it elsewhere.

**Note:** Downloads are limited to 4 GB max. If your study exceeds this, you can break it down into smaller chunks or use Tricefy Downlink. Downlink also allows multiple studies to be downloaded at once.

[Instructions for installing Tricefy Downlink](#)