

## Downloading Individual Images, Clips & Reports

Last Modified on 2026-03-18 11:32

### Downloading Individual Images or Files

Individual images, clips, reports, etc., can be downloaded by clicking on the dropdown menu in the upper left corner of any thumbnail (next to the capture number), then choosing the file type for download.

DCM to download as a DICOM file, JPG to download as an image file which can be viewed anywhere, MP4 to download cines as a video file playable on almost any device.



Note: Thumbnails are shown on the Study List, as well as the viewer (if thumbnails are enabled).

### Downloading Reports/PDFs in the Viewer

When a PDF document is opened in the Tricefy viewer, a download icon becomes available when the mouse is over the document (shown below in the red box):

