


## Downloading Individual Images, Clips & Reports

Last Modified on 05/23/2019 1:56 pm EDT


### Downloading Individual Items

Individual images, clips, reports, etc., can be downloaded by [opening the thumbnails for a study](#), selecting the upper-left corner of the thumbnail and choosing a file image (JPG to download a picture, MP4 to download a clip, etc.):


Downloading PDF document

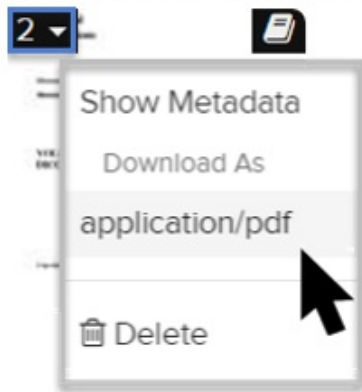

Downloading Image

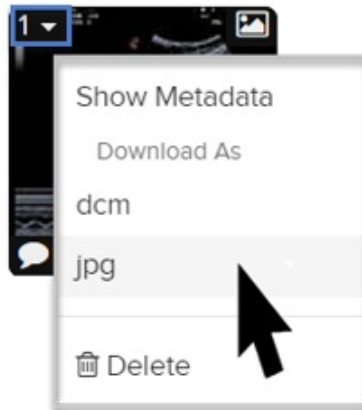


Downloading Cine Clip

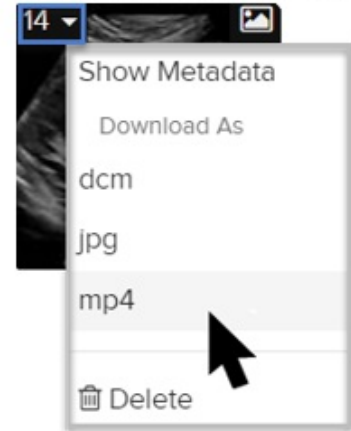

*Note: Thumbnails are shown on the Study List, as well as the viewer ( if thumbnails are enabled ).*

### Downloading Reports/PDFs in the Viewer

When a PDF document is opened in the viewer, a download icon becomes available when the mouse is over the document (shown below in the **red** box):

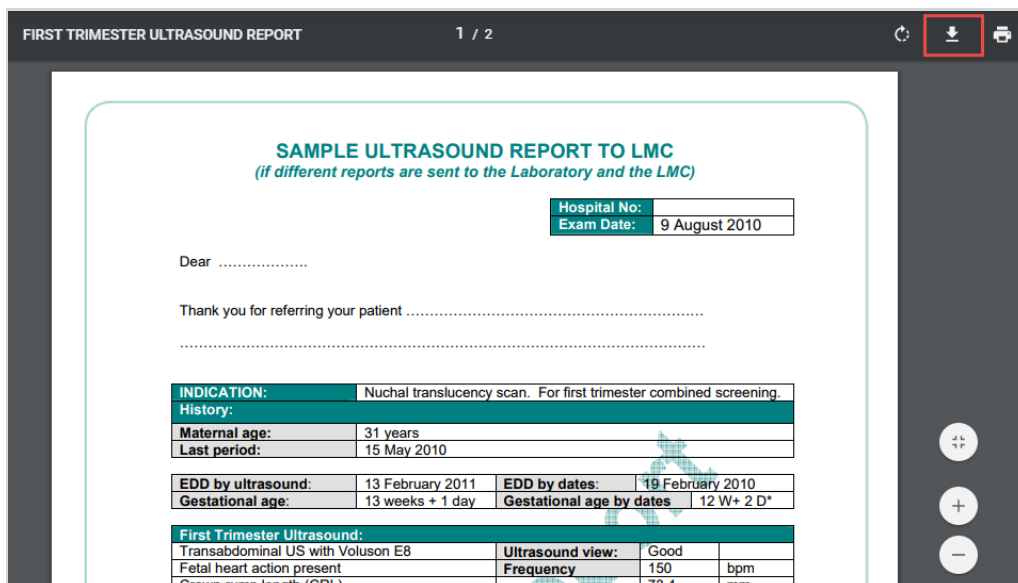

**FIRST TRIMESTER ULTRASOUND REPORT** 1 / 2

**SAMPLE ULTRASOUND REPORT TO LMC**  
*(if different reports are sent to the Laboratory and the LMC)*

Hospital No:	
Exam Date:	9 August 2010

Dear .....

Thank you for referring your patient .....

<b>INDICATION:</b>	Nuchal translucency scan. For first trimester combined screening.
<b>History:</b>	
<b>Maternal age:</b>	31 years
<b>Last period:</b>	15 May 2010

<b>EDD by ultrasound:</b>	13 February 2011	<b>EDD by dates:</b>	19 February 2010
<b>Gestational age:</b>	13 weeks + 1 day	<b>Gestational age by dates:</b>	12 W+ 2 D*

<b>First Trimester Ultrasound:</b>	
Transabdominal US with Voluson E8	<b>Ultrasound view:</b> Good
Fetal heart action present	<b>Frequency</b> 150 bpm
Crown-rump length (CRL)	73.4 mm