



Mac OS Uplink Installation

Last Modified on 2025-05-06 11:23


1. Uplink Installation

Newer versions of Mac OS may require Tricefy Uplink to be allowed to install and run.

The following error may display when attempting to install Tricefy Uplink:



1. Download Uplink as normal (https://downloads.tricefy.com/tricefy_uplink/uplink4.pkg)
2. Go to *System Settings*
3. Go to the *Privacy and Security* settings
4. Scroll to bottom and select *Open Anyway*

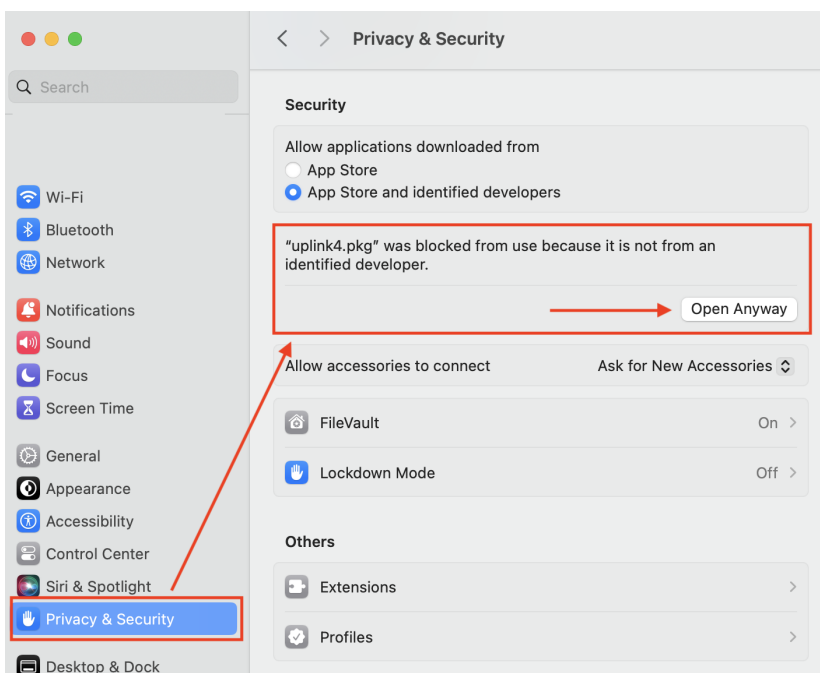

2. Uplink Firewall Settings (if needed)

If the ultrasound cannot ping/verify with the Mac uplink computer, you will likely need to add/allow the Tricefy uplink services to the Mac firewall settings.

1. *Network > Firewall > Options/Advanced > Add* (plus sign icon)
2. Navigate to *Applications > Tricefy_Uplink > bin*
3. Add **"boxComm"** and **"storescp"** to the list with the "Allow incoming connections" setting

