Treefy Mac OS Uplink Installation

Last Modified on 2025-06-30 11:43

1. Uplink Installation

Newer versions of Mac OS may require Tricefy Uplink to be allowed to install and run.

The following error may display when attempting to install Tricefy Uplink:



- 1. Download Uplink as normal (https://downloads.tricefy.com/tricefy_uplink/uplink4.pkg)
- 2. Go to System Settings
- 3. Go to the *Privacy and Security* settings
- 4. Scroll to bottom and select Open Anyway

•••	< > Privacy & Security
Q Search	Security
ᅙ Wi-Fi	Allow applications downloaded from App Store App Store and identified developers
🛞 Bluetooth	
🛞 Network	"uplink4.pkg" was blocked from use because it is not from an identified developer.
Notifications	Open Anyway
C Focus	Allow accessories to connect Ask for New Accessories 🗘
Screen Time	
Seneral	A selidava Mada
O Appearance	
() Accessibility	
Control Center	Others
Siri & Spotlight	Extensions >
Privacy & Security	Profiles >

2. Uplink Firewall Settings (if needed)

If the ultrasound cannot ping/verify with the Mac uplink computer, you will likely need to add/allow the Tricefy uplink services to the Mac firewall settings.

- 1. Network > Firewall > Options/Advanced > Add (+ icon)
- 2. Navigate to *Applications* > *Tricefy_Uplink* > *bin*
- 3. Add "boxComm" and "storescp" to the list with the "Allow incoming connections" setting

Trcefy

< > Fire	wall
 Antipit 	A file losting searches (
Citeria fatigate ante	 Maximum providence 2.
B	 How incoming connections: 2
 factives 	 Maximum providencial
Contractor and an	· the institution of the line of
A reasonable and the second	 Most function second local 2
storescp	Allow incoming connections \$
•	A film institute structure 3.
	a the territy provides 2
 manuf home as 	 More incoming connections - 2
4 miles	A the burning sensitive 3
Real of Street second	 Maximum providence 2
manipulated and	A first increase second and 2
and inspect	 Maximum providence 2
Contraction of the	 How incoming connections: 2
	 Maximum president 3
+ -	
	Cancel OK

2b. Place DICOM dictionary into Tricefy uplink folder (if needed)

After installing and activating the Tricefy Uplink, the DICOM dictionary file may need to be manually copied into the Tricefy uplink's asset folder.

- 1. Download the Tricefy DICOM dictionary file here: https://downloads.tricefy.com/tricefy_uplink/dicom.dic
- 2. Copy the file into the folder ~/Applications/Tricefy_Uplink/config
- 3. Restart the Uplink
 - a. Open the "UplinkSettings" program in the Applications list
 - b. Click the "Apply Settings" button (no need to make any changes)
 - c. Enter Mac user password if prompted to restart Uplink