



Worklist

Last Modified on 2021-03-08 17:28

 For athenahealth users, visit tricefy.help/help/athenaworklist

You can use your ultrasound system to query patient information using Worklist.

To setup Worklist, add a new DICOM Service:

1. Locate the DICOM Configuration screen (sometimes also called the "Device" or "Device Setup" screen)
2. Select the **Add** or **New** button and choose **Worklist** from the Services drop-down menu
3. Enter the following information:
 - AE Title = Worklist
 - IP Address = Enter the IP address of the Uplink computer *
 - Port = Enter the installation port (often 104 or 105) *

**You can view the IP address and installation port within your Uplink Account Settings

4. Select **Save**

Worklist Results

Depending on your modality worklist integration with Tricefy, the following data can potentially auto-populate into your ultrasound system, as long as the data was included in the order. This will prevent needing to manually enter information:

- Patient Name
- Patient ID
- Patient Birth date
- Patient Sex
- Patient Address
- Patient Phone
- Referring Physician
- Performing Physician Name
- Scheduled Performing Physician Name
- Study Description / Exam Time
- Scheduled Procedure Start Date
- Scheduled Procedure Start Time

Patients Screen Settings

Select the cog in the upper-right corner of the Patients screen to access the page settings and choose **Worklist** to display a worklist column:

	Full Name	Patient ID	Studies	Created Date & Time	Updated Date & Time	Show in Worklist
TD	Dean Gerald	123456	Show Studies (5)	2019-06-17 2:54 PM MST	2021-02-18 10:36 AM MST	<input checked="" type="checkbox"/>
TD	Dog Mallina	000001	Show Studies (1)	2017-06-28 8:46 AM MST	2021-02-10 2:43 AM MST	<input checked="" type="checkbox"/>
TD	Felicity Armstrong	9999999	Show Studies (1)	2018-05-10 9:47 AM MST	2021-01-28 6:21 AM MST	<input type="checkbox"/>
TD	SPENCER NANCY	AVI TEST	Show Studies (1)	2018-06-02 9:37 AM MST	2020-12-21 8:34 AM MST	<input type="checkbox"/>
TD	Audree Three	5d0e46e1-90e3-4249-9...	Show Studies (1)	2017-08-02 9:38 AM MST	2020-07-16 7:36 AM MST	<input checked="" type="checkbox"/>
TD	Westin Martin	68327	Show Studies (1)	2018-05-30 2:28 PM MST	2020-05-05 7:45 AM MST	<input type="checkbox"/>

Results will only return for patients with a checkmark in the worklist column.