



Default and Strict Anonymization


Last Modified on 2021-03-18 16:52

Tricefy is a fast and easy way to share medical images with patients while ensuring the data remains secure and anonymous.

Before sharing an exam with a patient, Tricefy Uplink automatically removes identifiable patient information from images. This includes the patient's name, birthday, gender, and doctor. For additional security, clinics have the option to require a password for patients to view their images.

While all data stored in Tricefy is encrypted, Trice also offers Strict Anonymization.

Strict anonymization removes all identifying information *before* uploading data to Tricefy. While most things, like patient address, weight, performing physician, study ID, procedure, etc. are completely removed, there are a few things that cannot be removed. These things are anonymized instead:

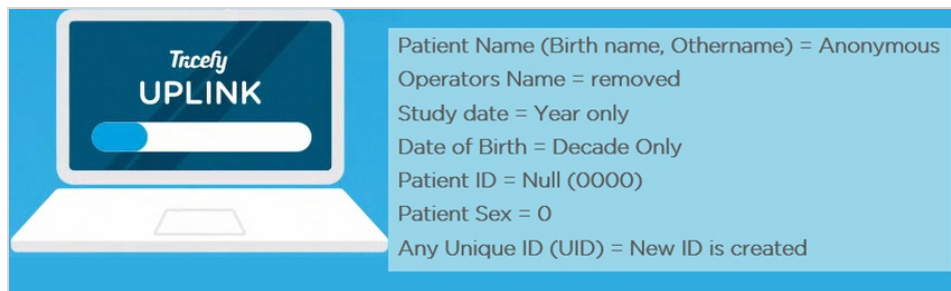

This means:

- Every patient's ID will be 0000
- Every patient's sex will be 0
- Dates (exam date, birthdate, etc.) will only display the decade (1900, 2000)
- All unique IDs will be newly generated without any traceability to an individual

This will make every examination in Tricefy non-distinguishable.

PATIENTS		STUDIES		CONSULTS						
		Full Name	Patient ID	Patient DOB	Gender	Procedure	Study Created	Source	Sonographers	Perf. Phys
TD	▶	ANONYMOUS	000000	1990	0		2017			
TD	▶	ANONYMOUS	000000	1990	0		2017			
TD	▶	ANONYMOUS	000000	1990	0		2017			

[See all the removed and anonymized data](#)