

Templates - Creating and Modifying Smart Groups

Last Modified on 2021-05-06 14:44

Smart Groups are grouped elements that can be inserted into other templates.

Creating Smart Groups

To create a Smart Group:

1) Open a template that contains the elements you would like to group

2) Select the items on the left menu

- Windows: Hold Shift to select multiple items
- Mac: Hold Control to select multiple items

3) Select Create Smart Group

4) You will be prompted to name your Smart Group. Choose a name that describes the grouping so you can easily identify it later.

2. SECOND AND THIRD TRIMESTER	2. Limited with MVP 🖌							+ Create Smart Group
T radio buttons	+ ~	2						La
8≡ foo								
D horizontal-rule		radio buttons	foo bar					
row			O baz					INSI
${f T}$ Limited With MVP			Limit	ed With	MVP			+
section	Γ	Patient	Patient Full Name	1	DOB:	Patient DOB	~	ARF
row		Patient ID Physician	Patient ID Empty	•	Exam date: Sonographer	Study Exam Date Empty	ž	
container		Est. EDD	📕 EDD from LMP 🗸 🗸					TEX
T Patient		GA by EDD	Gestational Age by EDD					De
T text		Indication(s) for to ultrasound:	day's Snip				11	
empty		Fetal Presentation	Empty	✓ Cardia	ac Activity Empty	~		ELE
container								88
T DOB:		radio buttons	 foo bar 					Bac
🛱 Date & Time			O baz					PAD
	l	radio buttons	foo					

Inserting Smart Groups

To insert a Smart Group:

1) Select + on the template where you would like to insert the grouped elements

2) Choose the Smart Group from the drop-down menu:



+ ~		
Section		
${f T}$ Text	>	
£ Forms	>	
Empty	>	
🗲 Templates	>	
${f A}$ Smart Groups	>	Filter smart groups.
N		Header and address
5	>	Help Center Article

Modifying Smart Groups

Smart Groups are indicated on the left menu. Selecting the word "Smart Group" will select the Smart Group on the template:

Container	Patient Patient
T Patient Name	h
T text	DOB: 🗸
A smart-group	+ ~

The blue lock indicates that any changes made to this Smart Group will occur to all templates using this Smart Group (the group is "linked").

Editing a Smart Groups

By default, Smart Groups are linked. To make changes that will affect all instances where the Smart Group is used, select Edit.

Patient			SMART GROUP		
Patient			Help Center A	rticle	•
DOB:	~		(AP)	<u>\$</u> 3	
+ ~			Edit	Unlink	
			INSERT		

When you are finished making changes, select Save in the upper-right hand corner:

There are ch	nanges wai	iting to be	saved.	Save	
There are ch	hanges wa	iting to be	saved.	Save	

To make changes that only affect this template, select **Unlink**. Unlinking the account will remove the blue lock and you can proceed to edit the elements directly on the template.

Managing Smart Groups

To edit or delete Smart Groups, select Manage Smart Groups from the top menu:



						+ Create Smar	rt Group 💠 Preview as 🗸 🚺
	Heading with	Underline					T: Manage Smart Groups T: Manage Text Snips
	JUst Patient I Label snip	D (Text) Text		Patient Age Only Number)			Export Template
+ ~	Fetal Anatomy	Snip					Fetal Heart Rate 💉 📦
	Patient	Patient Full Name	11	DOB:	🎽 Patient DOB	~	+ Add More
	Patient ID Physician	Patient ID Empty	v //	Exam date: Sonographer	Study Exam Date Empty	*	ARRANGEMENT
	Est. EDD	🗡 EDD from LMP 🗸 🗸					Columns Move
	GA by EDD	Gestational Age by EDD					TEXT
	Indication(s) for to ultrasound:	day's Snip					Default V B I
	Patient	Patient Full Name	11	DOB:	🗡 Patient DOB	~	
	Patient ID Physician	Patient ID Empty	•	Exam date: Sonographer	Study Exam Date Empty	* *	

A pop-up window allows you to select any Smart Group. It does not have to be used in the current template:



- Select Edit to view and make changes to the Smart Group
 - When done modifying the Smart Group, Select Save in the upper-right corner
- Select the trash icon to remove the Smart Group