



## Importing archived data

Last Modified on 2026-07-09 12:28

You can import large amounts of data for archiving:

1) Contact your sales representative or our support center ( [support@triceimaging.com](mailto:support@triceimaging.com)) and let us know:

- Total data size
- Number of studies or files\* to import

\* Only DICOM files are accepted (non-DICOM files will not import)

2) A quote will be provided for the service of importing and archiving the data

3) A Trice representative will walk through installing [Tricefy Uplink](#) software

4) The Trice team will work with the customer to ensure the Tricefy Uplink upload process is working, and suggest the how to upload the data in batches

5) The customer will continue to upload the archived DICOM files in batches via the Tricefy Uplink

Once the DICOM data is uploaded it can be searched and viewed immediately in the Tricefy web portal

---