


Canon Applio and Xario Series Configuration Instructions


Last Modified on 2023-07-27 23:55

Follow the process below to configure your ultrasound system to send data to Tricefy.


1. First, go to Configuration button on the side of the touch panel, followed by the Service button:

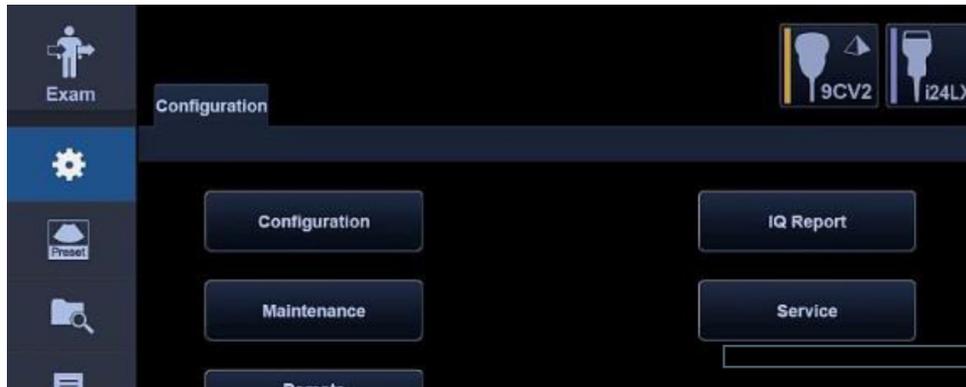

2. Enter access information - you must have a service user ID and password to configure the ultrasound to proceed

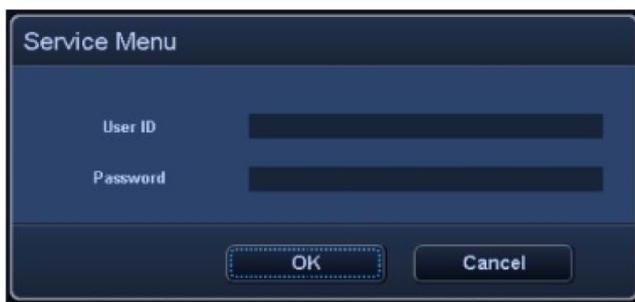

3. Once the system has been connected to a network, go to Config - Network Settings - Click Edit

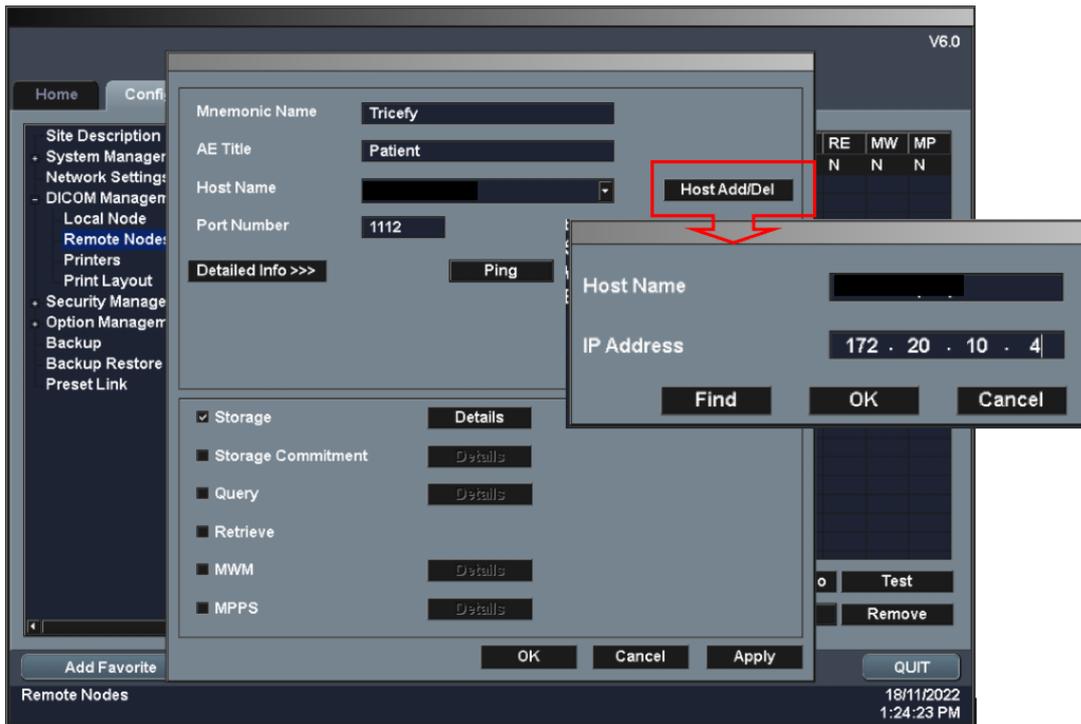

Mnemonic: Tricefy

AE Title: patient


Click Host Add/Del and create Host Name

IP Address = shown during installation or refer to [Account Settings](#)

Port = 1112 (refer to [Account Settings](#))

9. Mark Storage, click Details

10. Set the Compression Type as shown - then Select Advance 1 and uncheck all boxes.

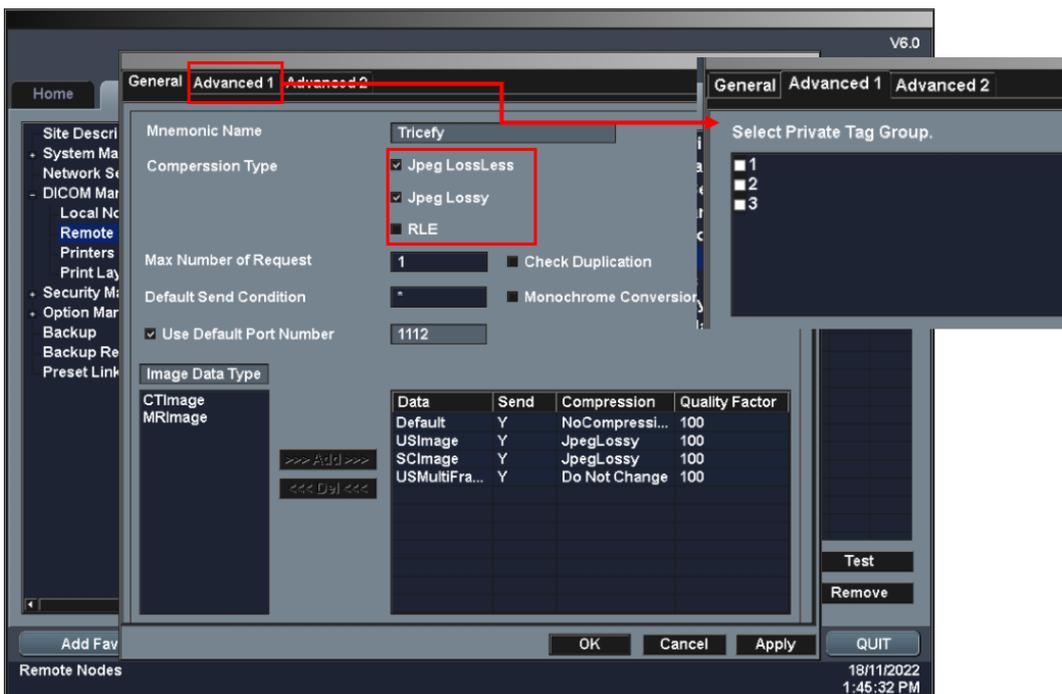



11. Repeat steps 8-10 for adding the Storage function by enter following information

Mnemonic: Tricefy

AE Title: archive

Click Host Add/Del and select created host.

IP Address = shown during installation or refer to [Account Settings](#)

Port = 1112 (refer to [Account Settings](#))

12. Press Apply and OK.

13. Verify the connection with Ping - DICOM Echo - Test

14. To finalize the setup open System Preset - Device Setting - Printer & Server Assigned

