



Mindray TE Air

Last Modified on 2024-12-04 15:02

Connecting your TE Air app to Tricefy is very quick and easy. Using the "Tricefy Send" option built into the TE Air app is the preferred method to send studies to Tricefy cloud as it will not require a separate app on the device.

Please follow the instructions below to connect your TE Air app to Tricefy on any iOS or Android device.

1. Generate API token in the Tricefy web portal

Important: you must be an **admin** of the Tricefy account in order to generate an API token

1. Log into the Tricefy web portal: <https://tricefy4.com>
2. Go to the [Tricefy account settings](#) and select the Tricefy account you wish to activate the TE Air under
3. On the left hand side choose the **API Tokens** option near the bottom of the list
4. Click the **Issue Token** button
5. Give the API token a helpful name (e.g. TE Air 1 - POC)
 - Note: it is strongly recommended to generate [1 API token for each TE Air device](#).
6. Write down or copy/paste the API token displayed on the screen as you will need it for the next steps below with the TE Air app
 - Note: the API token will be only be displayed on the create token screen once. After you leave the create token screen you will not be able to view the token again


Token | My TE Air

<API_token_displayed_here>

- Tokens are only diplayed once!


2. Send to Tricefy on the TE Air app

1. Within the TE Air app, tap the **Menu** icon in the top left of the screen
2. Select "iStation" from the menu

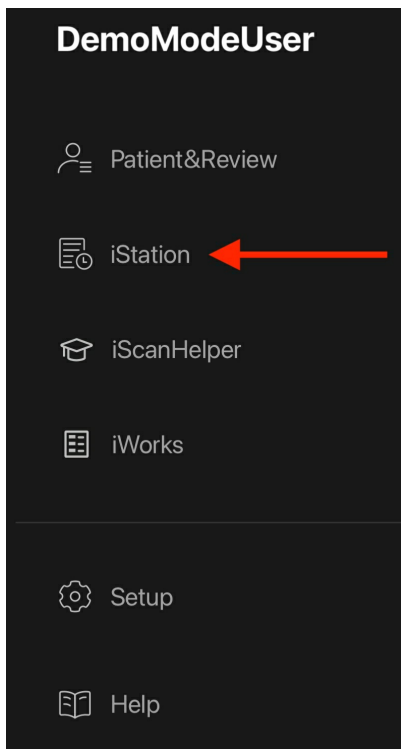

3. Select 1 or more studies from the list of studies

4. Tap the "Send To" button near the bottom of the screen and choose "Send to Tricefy"

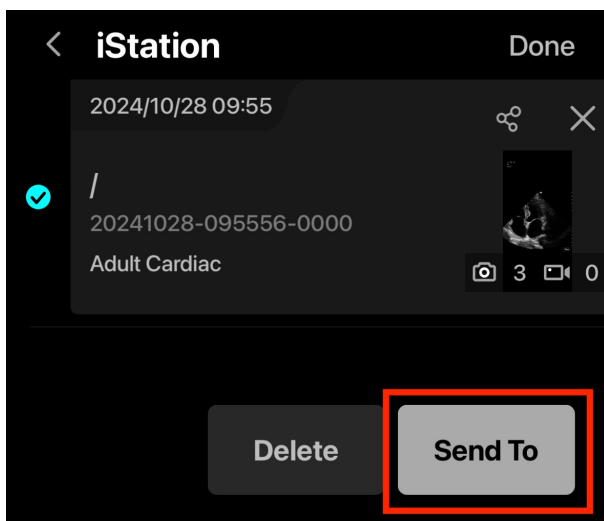

5. In the "Send to Tricefy" pop up, enter your Tricefy login email into the username field, and the [API token](#) you generated from the earlier step above into the password field

Send to Tricefy

Username:
my_tricefy_login_email@domain...

Password:
_____ **Tricefy**

Cancel **OK**

6. Click "OK" to send to Tricefy
